



Maintenance/Storage

Flush the manikin with 1/2 liter of water and drain.

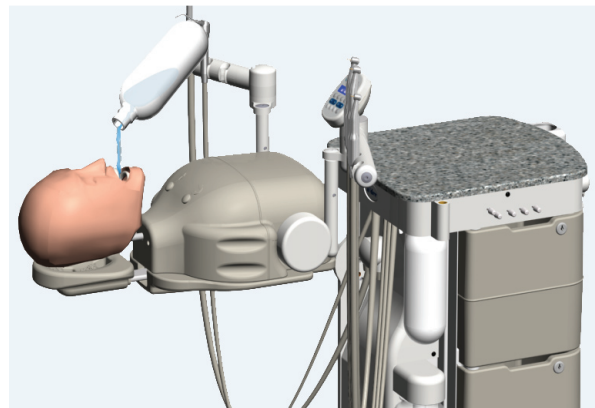

Empty the optional vacuum screen.

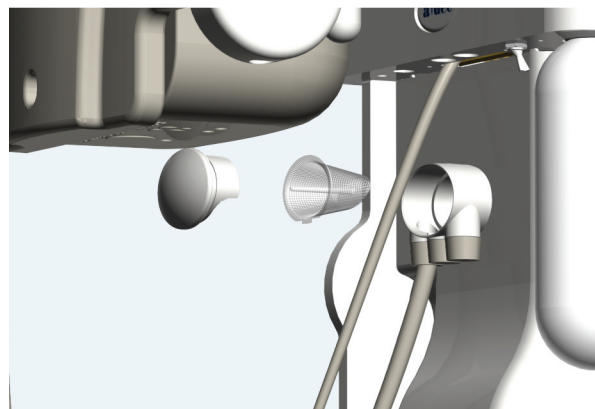

Note: Clean the Simulator unit with warm water and soap. Do not use any chemicals or cleaners.

Regulatory Information


Regulatory information is provided with A-dec equipment as mandated by agency requirements. This information is delivered in the separate *Regulatory Information and Specifications* document. If you need this information, please go to the Document Library at www.a-dec.com.




CAUTION: Before plugging in any external device, note that the optional electrical outlet has an output rating of 100-120 VAC at 7 Amps maximum.



Empty the optional water bottle and vacuum waste jar after use.

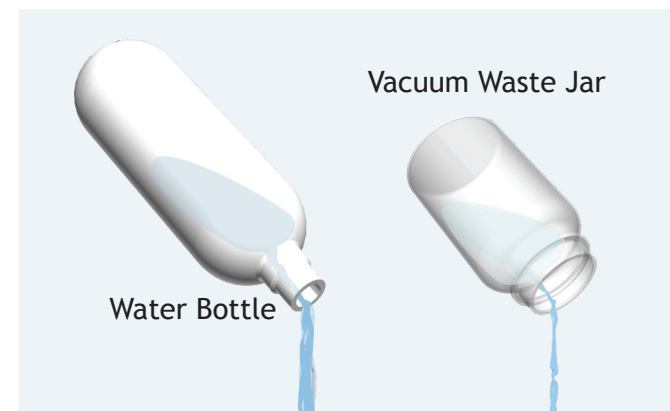

Return the unit to its storage position.

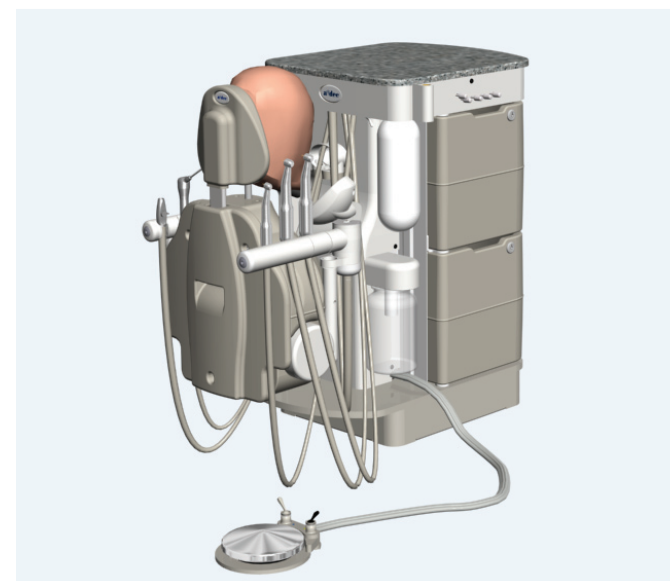

Turn the unit off.



A-dec Simulator Setup, Use, and Maintenance




This product is intended for instructional use in a laboratory setting.

Setup


Turn the unit on.




Press the Rotation button to tilt the torso.

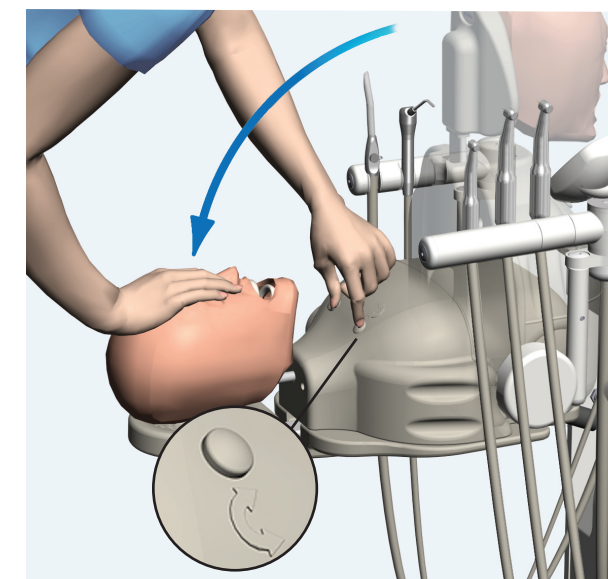

Press the Up/Down button to raise the torso.

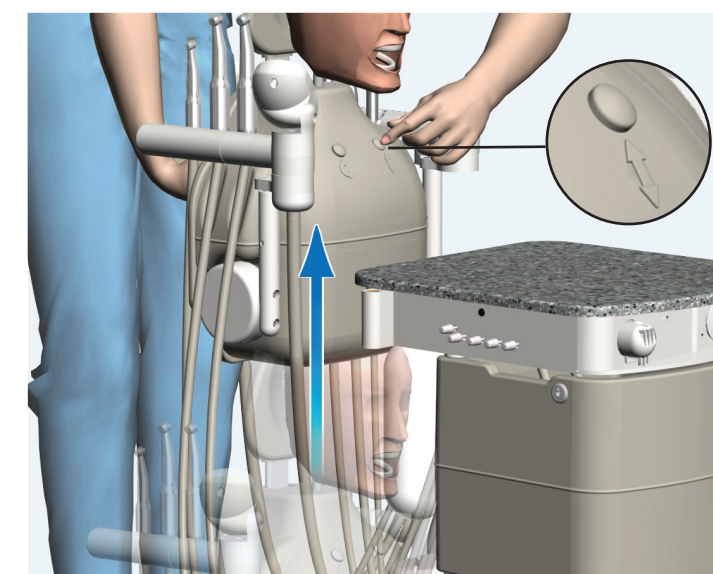


Swing the holder arms out.

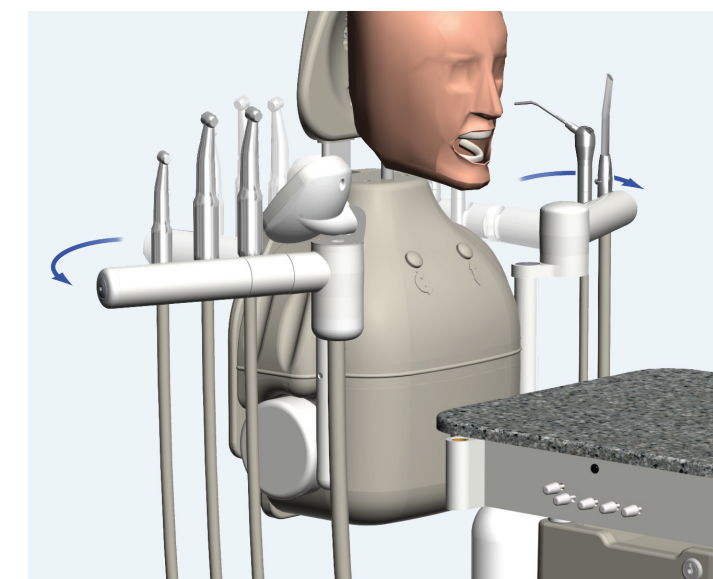


Setup (continued)

Fill the optional water bottle.

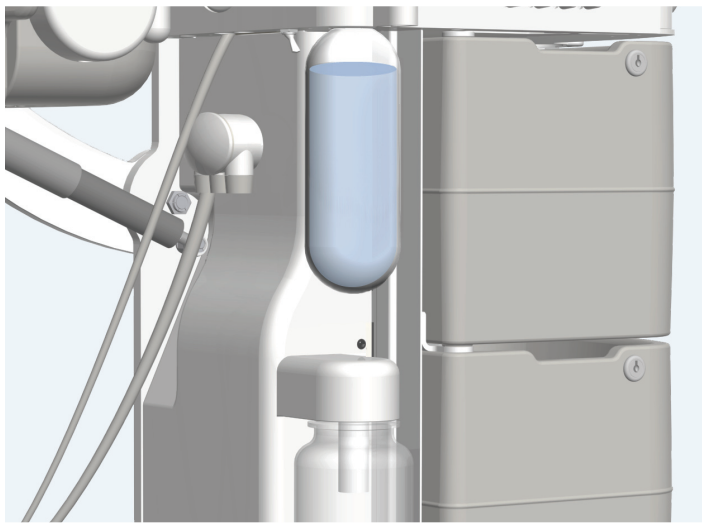


Insert the syringe tip until two clicks are felt, indicating the syringe tip is properly installed.

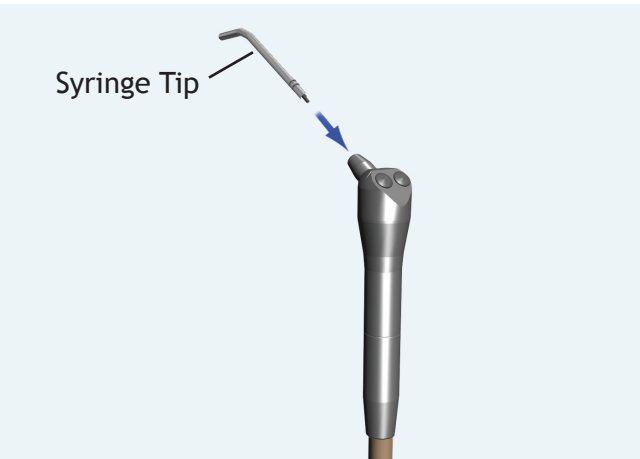

WARNING Use only A-dec syringe tips to prevent ejection. When installing a tip, be sure to insert the tip fully.


Use

Adjust the air and water for handpieces.

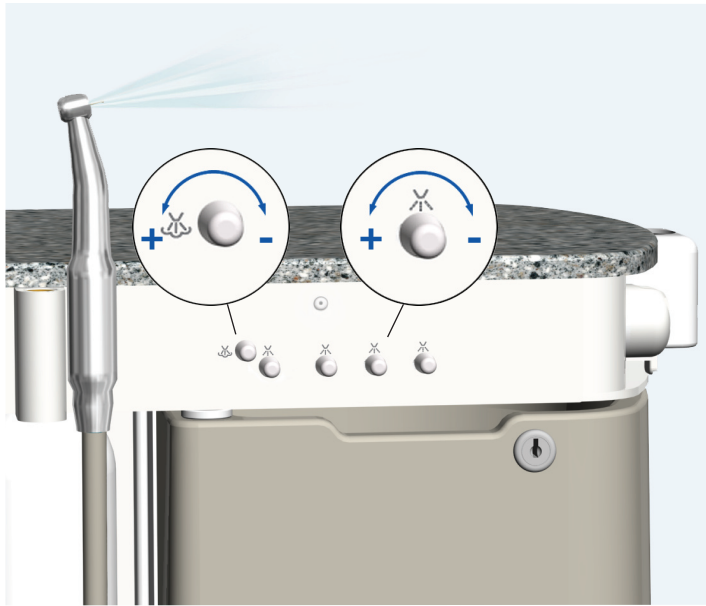


CAUTION Do not turn the key all the way to the left or the stem could come out of the block.

Lift out the holders and switch sides to convert for right- or left-handed use.

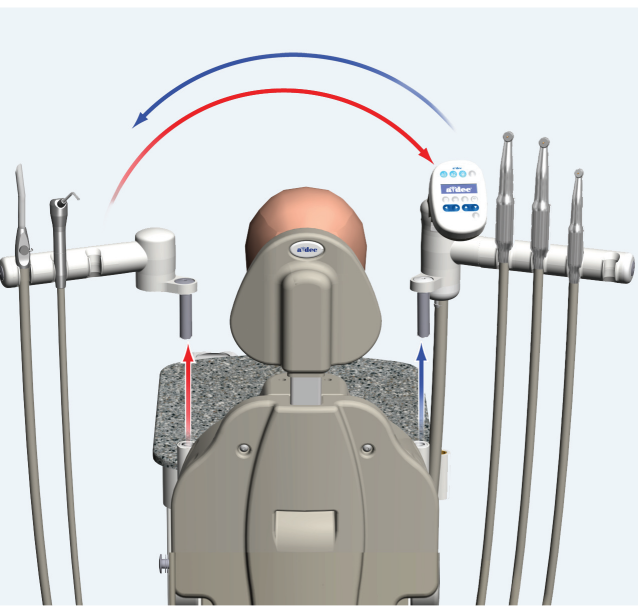

Use (continued)

Use the foot control to operate the vacuum and handpieces.

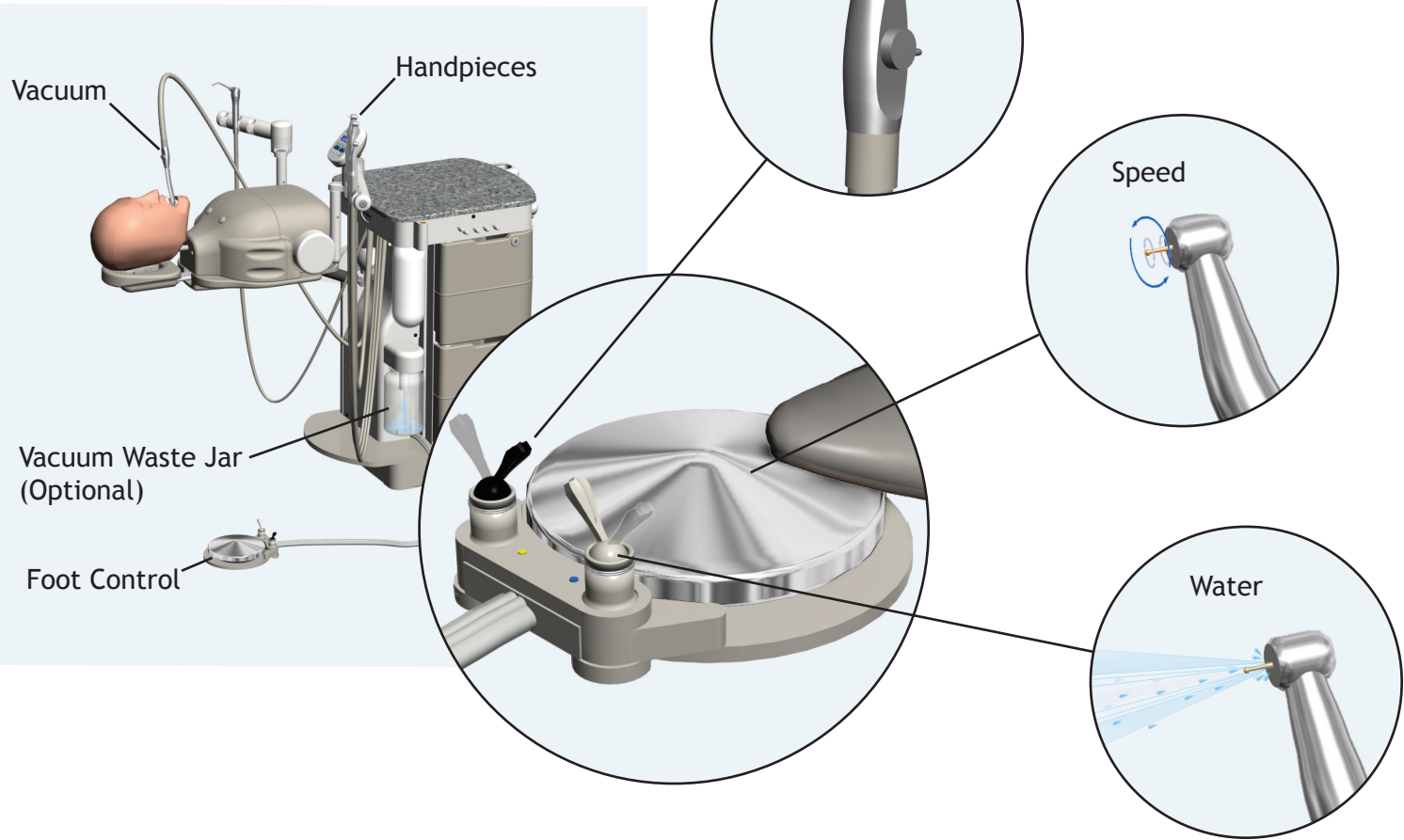

Use the touchpad to adjust the Simulator handpieces or operate the dental light. (Touchpad is optional. Not all features shown on the touchpad may be available.)

