

# Product information

## FD 366 sensitive top wipes

### disinfection of sensitive surfaces



#### FD 366 sensitive top wipes at a glance

- Practical disinfecting wipes for quick-acting disinfection of sensitive surfaces on invasive and non-invasive medical devices in private practices, laboratories and hospitals
- Good material compatibility. Particularly suitable for sensitive surfaces, such as acrylic glass, plastics and synthetic leather coverings on treatment units
- Exposure time according to VAH only 1 minute
- Broad spectrum of activity: bactericidal, tuberculocidal, yeasticidal, virucidal activity against enveloped viruses (enveloped viruses incl. HBV, HCV, HIV, coronaviruses as well as non-enveloped noroviruses)
- FD 366 sensitive has been tested as the active substance solution in FD 366 sensitive top wipes in accordance with current VAH methods and European standards
- Tested in 4-field test in accordance with VAH and EN 16615
- FD 366 sensitive top wipes are VAH-certified. FD 366 sensitive is included in the VAH list and IHO list of disinfectants
- Wipe size: 175 x 250 mm

#### Properties

FD 366 sensitive top wipes from Dürr System Hygiene are aldehyde-free, alcohol-based ready-to-use wipes for quick-acting disinfection of sensitive surfaces on invasive and non-invasive medical devices and medical furnishings in private practices, laboratories and hospitals. FD 366 sensitive top wipes are distinguished by fast efficiency and good material compatibility. Disinfected surfaces dry rapidly.

#### Product composition

FD 366 sensitive top wipes contain FD 366 sensitive, a quick-acting combination of alcohols and quaternary ammonium compounds as well as excipients. 100 g active substance solution contains 17 g 1-propanol, 0.125 g alkyl benzyl dimethyl ammonium chloride, excipients and water.

#### Microbiological efficacy

FD 366 sensitive top wipes are bactericidal<sup>1)</sup>, tuberculocidal<sup>1)</sup>, yeasticidal<sup>1)</sup>, virucidal activity against enveloped viruses (enveloped viruses incl. HBV, HCV and HIV<sup>2), 3)</sup> as well as non-enveloped noroviruses<sup>4)</sup>. FD 366 sensitive is included in the VAH list and IHO list of disinfectants. The active substance solution has been tested in accordance with DVV/RKI guidelines, EN 13727, EN 13624, EN 14348, EN 14476, EN 16615. FD 366 sensitive top wipes are tested as ready-to-use wipes in 4-field test in accordance with VAH/EN 16615.

#### Directions for use

Remove FD 366 sensitive top wipes individually. Wipe the surfaces and objects to be disinfected and allow the exposure time to elapse. Disinfect visibly contaminated surfaces with FD 366 sensitive top wipes and remove contamination. Disinfect cleaned surfaces again. Make sure that wetting is complete throughout the entire exposure time. Reseal the flow pack immediately after removing a wipe. In accordance with VAH, the exposure time is 1 minute, for Tb bacteria 2 minutes and for viruses 30 seconds to 5 minutes.

# Product information

## FD 366 sensitive top wipes

### disinfection of sensitive surfaces

#### Environmental impact

The packaging consists of a composite material and can be incinerated. Dispose of used wipes with the domestic waste. Do not flush.

For further information see safety data sheet.

#### Physical data

##### Active substance solution:

Appearance: clear, colourless solution

Density:  $D = 0.97 \pm 0.02 \text{ g/cm}^3$  (20°C)

Flash point: 33°C (acc. to DIN 51755)

#### Shelf life

3 years

After opening, 1 month.

#### Container sizes

Flow pack with 50 wipes

4 x 50 wipes in the box

#### Storage

Store product at between 5°C and room temperature.

#### General instructions

Disinfect the exterior surfaces of medical devices, such as dental curing lights, nozzles, handpieces and contra-angle handpieces, prior to processing as required in accordance with the manufacturers' instructions.

In the case of sensitive surfaces, test for compatibility in an inconspicuous place. Changes in product colour/odour may occur, especially when stored in sunlight. However, these changes in colour and/or odour have no impact on the disinfection efficacy of the product. Use exclusively by qualified personnel. All serious incidents that occur in association with the product must be immediately reported to the manufacturer and the competent authority of your member state.

#### Sales

Dürr Dental SE  
Höpfigheimer Str. 17  
74321 Bietigheim-Bissingen  
Germany  
www.duerrdental.com  
info@duerrdental.com

#### Manufacturer

orochemie GmbH + Co. KG  
Max-Planck-Straße 27  
70806 Kornwestheim  
Germany  
www.orochemie.de  
info@orochemie.de

Application	Time
Surface disinfection (VAH methods) <sup>1)</sup>	1 min
Bacteria <sup>1)</sup> and yeasts <sup>1)</sup>	1 min
Tb bacteria <sup>1)</sup>	2 min
Enveloped viruses incl. HBV, HCV, HIV, coronaviruses <sup>2), 3)</sup>	30 sec
Noroviruses <sup>4)</sup>	5 min

1) Testing of the active substance solution at low and high load (VAH, EN 13727, EN 13624, EN 14348, EN 16615).

2) Testing with and without load in accordance with DVV/RKI guidelines.

3) In accordance with RKI statement (Federal Health Bulletin 60, 353 - 363, 2017).

4) Testing at low load (EN 14476).

#### Hazard warnings

FD 366 sensitive top wipes are classified and labelled according to the CLP Regulation: see product label and safety data sheet.

#### Independent expert opinions – in-house investigations

The expert opinions are available upon request.

**CE 0124**