

# Product information

## FD multi wipes green / FD multi wipes compact green

### Non-woven wipes



#### FD multi wipes green/FD multi wipes compact green at a glance

- Premium-quality, tear-resistant, lint-free, dry wipes in a refillable dispenser.
- The wipes are made from environmentally-friendly, plastic-free natural fibres.
- Only for use with working solutions of FD 312 4%, FD 322, FD 333 and FD 333 forte.
- Finely structured, nonwoven material with excellent wetting properties that won't scratch surfaces.
- Extra-large wipes sized 200 x 300 mm or 290 x 300 mm.
- Proven disinfecting action and stable solutions.
- Effective from the first day to the last – shelf life of 28 days.
- Practical, resealable dispenser for refilling.
- Highly economical – enormously time-saving: the disinfecting wipes only need to be soaked once, reducing disinfectant consumption. Sufficient for 180 or 60 disinfection procedures, respectively.

#### Properties

FD multi wipes green/FD multi wipes compact green from Dürr System Hygiene are premium-quality, tear-resistant wipes.

FD multi wipes green/FD multi wipes compact green are suitable only for soaking with the products FD 312 4%, FD 322, FD 333, FD 333 forte. The wipes are made from environmentally-friendly, plastic-free natural fibres. FD multi wipes green/

FD multi wipes compact green provide for excellent distribution of the disinfectant solution, ensuring the complete wetting of the surfaces to be disinfected. FD multi wipes green/FD multi wipes compact green are sized and made of materials that can be easily folded to reach surfaces that are small or difficult to access. Using FD multi wipes green/FD multi wipes compact green reduces preparation times for disinfection procedures and disinfectant consumption. The dispenser protects the soaked FD multi wipes green/FD multi wipes compact green against contamination and evaporation.

#### Product composition

FD multi wipes green are sized 200 x 300 mm.

FD multi wipes compact green are sized 290 x 300 mm.

#### Directions for use

Use FD multi wipes green/FD multi wipes compact green to clean and disinfect surfaces on treatment units, work surfaces, walls, medical equipment, and objects of all kinds. Prepare FD 312 as a 4% active substance solution. FD 322, FD 333 and FD 333 forte are ready-to-use solutions that are applied undiluted. Wear gloves. Soak the wipes evenly with 2.5 L solution in the FD multi wipes green dispenser or with 1.4 L solution in the FD multi wipes compact green dispenser, thread the top wipe, seal the dispenser tightly, and wait 30 to 60 minutes before using the first wipe. Enter the product name, concentration, batch number, production date, expiry date, and name of the responsible person on the label enclosed with each roll of wipes and adhere the label to the resealable lid on the dispenser. Remove a wipe and use it to completely wet the surface to be disinfected. Allow the exposure time of the disinfectant in use to elapse as described in the instructions for use. Reseal the dispenser tightly after each use. Before refilling or replacing the product,

# Product information

## FD multi wipes green / FD multi wipes compact green

### Non-woven wipes

empty the dispenser of any remaining wipes and liquid, remove and dispose of the label from the lid, and process the dispenser. Manual processing: Manually scrub and remove any deposits or residues. Thoroughly rinse the inside and outside of the dispenser, including the lid, with hot tap water (> 55°C - Caution: risk of scalding!) and dry with a paper towel. Then, using a disposable wipe with alcoholic fast-acting disinfectant completely wet and thoroughly disinfect dispenser box and lid (allow sufficient time of action!). Make sure to dry the dispenser and lid completely. Alternative automated processing: Process and dry the dispenser box and lid in a chemothermal cleaning and disinfection system at up to 93°C. Place a new roll of wipes in the clean and dry dispenser and prepare as described above: soak the wipes, thread the top wipe, seal the dispenser tightly, adhere a new label to the lid, and wait 30 to 60 minutes before using the first wipe. Do not process contaminated dispenser boxes, dispose of them. Freely protruding wipes may be contaminated and must be discarded. Do not place removed wipes back into the dispenser. Do not use wipes that have dried out. Replace defective dispenser boxes.

#### Environmental impact

The packaging film is made from renewable raw materials. Dispose of used wipes with the domestic waste. Do not flush.

#### Shelf life

Use soaked wipes within 28 days months. For more detailed information, see the summary of product characteristics for the products on our website.

#### Package sizes

##### FD multi wipes green:

Dispenser with 1 roll of 180 wipes.

Refill pack with 4 rolls of 180 wipes each in the box.

##### FD multi wipes compact green:

Dispenser with 1 roll of 60 wipes.

Refill pack with 8 rolls of 60 wipes each in the box.

#### General instructions

Use exclusively by qualified personnel. All serious events that occur in association with the product must be immediately reported to the manufacturer and the competent authority of your member state.



#### Sales

Dürr Dental SE  
Höpfigheimer Str. 17  
74321 Bietigheim-Bissingen  
Germany  
www.duerrdental.com  
info@duerrdental.com

#### Manufacturer

orochemie GmbH + Co. KG  
Max-Planck-Straße 27  
70806 Kornwestheim  
Germany  
www.orochemie.de  
info@orochemie.de