**A-dec ICX Renew Solution 1 and Solution 2** is a specially formulated dental unit waterline shock treatment. It is intended to lower bacterial contamination in effluent and remove buildup of non-pathogenic microbial contamination from the waterlines. This ready to use two-part product cleans dental unit waterlines and removes the buildup of odor-causing, foul tasting bacteria. Always use ICX Renew Solution 1 and 2 together. Do not add water. This product is to be used in conjunction with regular testing of dental unit water.

**KEEP OUT OF REACH OF CHILDREN**

**CAUTION**

**PRECAUTIONARY STATEMENTS. Solution 1 and 2:**

**Hazards to Humans and Domestic Animals**

**CAUTION.** Causes moderate eye irritation. Harmful if absorbed through skin. Avoid contact with eyes, skin, or clothing. Wear protective eyewear (goggles, face shield, or safety glasses) and rubber gloves when handling. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum, using tobacco or using the toilet.

**CAUTION.** Do not use with a water maintenance cartridge or straw.

**DIRECTIONS FOR USE. It is a violation of federal law to use this product in a manner inconsistent with its labeling.**

**Applying ICX Renew to a Dental Unit's Waterlines**

1. Turn off the delivery system master toggle; then remove and empty the water bottle.
2. Remove air/water syringe tips, handpieces, detachable motors and couplers, and ultrasonic instruments from their tubing.
3. Add one bottle (4 ounces) of ICX Renew Solution 1 and one bottle (4 ounces) of ICX Renew Solution 2 into the empty water bottle.
4. Cover the opening of the water bottle and swirl the solution, coating all surfaces of the water bottle.
5. Reattach the water bottle to the unit and turn on the master toggle.
6. Run ICX Renew through the system until the blue solution flows out of the air/water syringe(s) and all of the handpiece tubing. Leave remaining solution in the water bottle.
7. Turn off the master toggle and allow the solution to remain in the unit for 12 to 96 hours.

**NOTE** Place a cloth or paper towel underneath the handpiece tubing to catch potential drips.

8. After 12 to 96 hours, turn on the master toggle and flush remaining ICX Renew solution through all air/water syringe and all handpiece tubing.
9. Turn off the master toggle, remove the water bottle, discard any remaining solution, and thoroughly rinse the bottle with water.
10. Refill the bottle with water, reattach it to the unit; then turn on the master toggle.
11. Completely flush all air/water syringe and all handpiece tubing into a sink or cuspidor until the bottle is empty.
12. Add an ICX® tablet to the water bottle and refill it with water.

**IMPORTANT.** For **new equipment** apply one treatment before initial use. For **existing equipment**, apply three treatments when your water quality test results exceed your water quality action level. You may use the dental unit between treatments. For **preventive maintenance**, you may apply one treatment on a routine basis.

## FIRST AID. Solutions 1 and 2:

Have product container or label with you when calling a poison control center or doctor, or going for treatment.

**If in Eyes:** Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after five minutes, then continue rinsing. Call a poison control center or doctor for treatment advice.

**If on Skin or Clothing:** Take off contaminated clothing. Rinse skin immediately with plenty of water for 15-20 minutes. Call a poison control center or doctor for treatment advice.

## STORAGE AND DISPOSAL. Solutions 1 and 2:

Do not contaminate water, food, or feed by storage disposal.

**Product Storage.** Keep the product in the closed original container when not in use. Store in a cool, dry area away from potential sources of heat, open flames, sunlight or other chemicals. Store between 5 °C (41 °F) and 30 °C (86 °F).

**Product Disposal.** Wastes resulting from the use of this product must be disposed of on site or at an approved waste disposal facility.

**Container Handling:** Nonrefillable container. Do not reuse or refill this container. Offer container for recycling if available or puncture and dispose of in a sanitary landfill, or by incineration.

## INGREDIENT STATEMENT. Solutions 1 and 2:

### Active Ingredients:

**Solution 1** - Hydrogen peroxide: 2.1%, Other ingredients: 97.9%, Total: 100%

**Solution 2** - Other ingredients: 100%, Total: 100%.

**Regulatory Information and Warranty:** For required regulatory Information and the A-dec warranty, see the Regulatory Information, Specifications, and Warranty document (p/n 86.0221.00) available in the Resource Center at [www.a-dec.com](http://www.a-dec.com).

A-dec, the A-dec logo, ICX, and ICX Renew are trademarks or registered trademarks of A-dec, Inc. in the U.S. and other countries.

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### **A-dec Headquarters**

2601 Crestview Drive  
Newberg, Oregon 97132  
United States  
Tel: 1.800.547.1883 within USA/CAN  
Tel: +1.503.538.7478 outside USA/CAN  
www.a-dec.com

### **A-dec Australia**

Unit 8  
5-9 Ricketty Street  
Mascot, NSW 2020  
Australia  
Tel: 1.800.225.010 within AUS  
Tel: +61.(0)2.8332.4000 outside AUS


### **A-dec China**

A-dec (Hangzhou) Dental Equipment Co., Ltd.  
Building 5, No.528 Shunfeng Road  
Tangqi Town, Yuhang District  
Hangzhou, Zhejiang, China 311100  
Tel: 400.600.5434 within CN  
Tel: +86.571.89026088 outside CN

### **A-dec United Kingdom**

Austin House  
11 Liberty Way  
Nuneaton, Warwickshire CV11 6RZ  
England  
Tel: 0800.233285 within UK  
Tel: +44.(0).24.7635.0901 outside UK

Donawa Lifescience  
Consulting Srl  
Piazza Albania, 10  
00153 Rome, Italy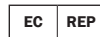

86.0741.00 Rev B  
Date of Issue 2020-10-27  
Copyright 2020 A-dec, Inc.  
All rights reserved