# VistaScan

Speicherfolien Plus

Image Plates Plus

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Placas radiográficas Plus

Film ai fosfori Plus

Opislagfolies Plus

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Bildplattor Plus

Fosforplader Plus

Κυναlevyt Plus

Fosforplater Plus

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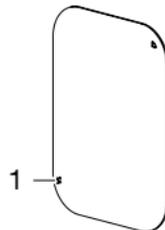
1 Hexadezimalcode

## General Information

Refer also to installation and operating instructions for VistaScan ....

EN

### Symbols

0297 CE labelling

Batch number

Manufacturer

### Intended purpose

The image plates are used to create intraoral and extraoral X-ray images.

### Intended use

The image plates are designed for use solely with Dürr Dental image plate scanners and Dürr Dental light protection covers in dental applications.

### CAUTION

#### Image plates are toxic

Image plates that are not packed in a light protection cover can lead to poisoning when placed in the mouth or swallowed.

- › Only place image plates in the patient's mouth in a light protection cover.
- › Do not swallow the image plate or parts of it.
- › If the image plate or parts of it have been swallowed, consult a specialist doctor immediately and remove the image plate.
- › If the light protection cover has been damaged in the patient's mouth, rinse the mouth thoroughly with lots of water. Do not swallow the water in the process.

- › Image plates are flexible like X-ray film. However, the image plates should not be bent.
- › Do not scratch the image plates. Do not subject the image plates to pressure from hard or pointed objects.
- › Protect the image plates against sunlight and ultraviolet light. Store image plates in a light protection cover or plate cassette of the correct size.
- › Image plates will be pre-exposed on exposure to natural radiation and stray x-ray radiation. Protect erased and exposed image plates from X-ray interference. If the image plate has been stored for longer than one week, erase the image plate prior to use.
- › Do not store image plates under hot or moist conditions.
- › When used properly, image plates can be exposed, read and erased several hundred times provided there is no mechanical damage. Replace the image plate if there are any signs of damage (e.g. protective layer is damaged or visible scratches) that could interfere with the diagnosis.
- › Image plates that have a production or packaging defect will be replaced by Dürr Dental in the same quantity. Claims can only be accepted within 7 working days after receipt of the goods.

### Classification

Medical device directive (93/42/EEC)..... Class IIa

### Storage and transport

Temperature (C°)..... < 33

Relative humidity (%)..... < 80

### Usage

Temperature (C°)..... 18 - 45

Relative humidity (%)..... < 80

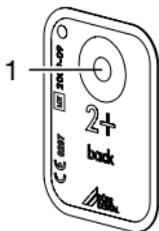
## Image Plates Plus ID

Prior to every use make sure that the marker adheres correctly. If the marker does not adhere correctly, do not use the image plate any more.


### Exposure from the wrong side

A marker is attached to the inactive side of the image plate.

1 Marker

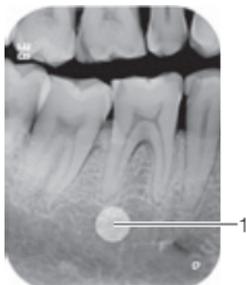If the image plate has been exposed from the wrong side, the marker is visible as a shadow in the X-ray image.

1 Marker visible as a shadow

The image can be corrected by mirroring it in the software. If a diagnosis is not possible in the area of the marker then the image will need to be acquired again.


With the aid of the copper points set you can retrospectively add a marker to image plates.

### Clear assignment of image plate to image (Image Plate Plus ID only)

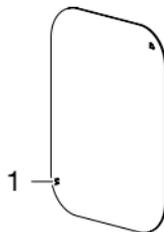On the Image Plate Plus ID there is also a hexadecimal code on the image plate in addition to the marker. This code can be seen in the X-ray image.

This code allows you to clearly assign the correct image plate to the X-ray image.

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