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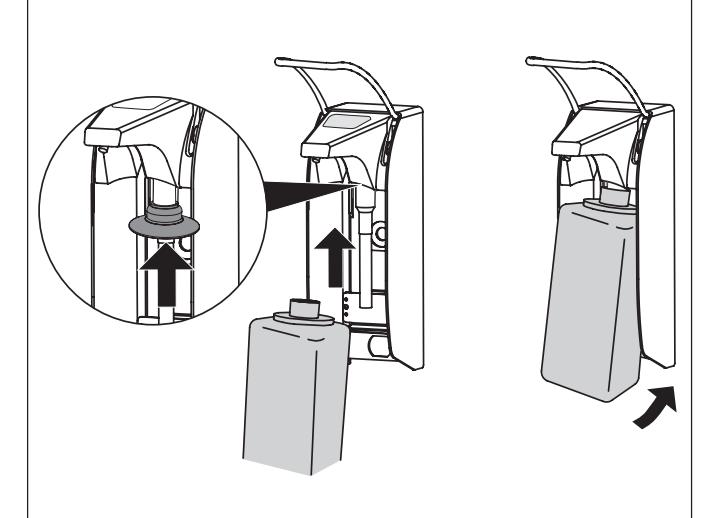
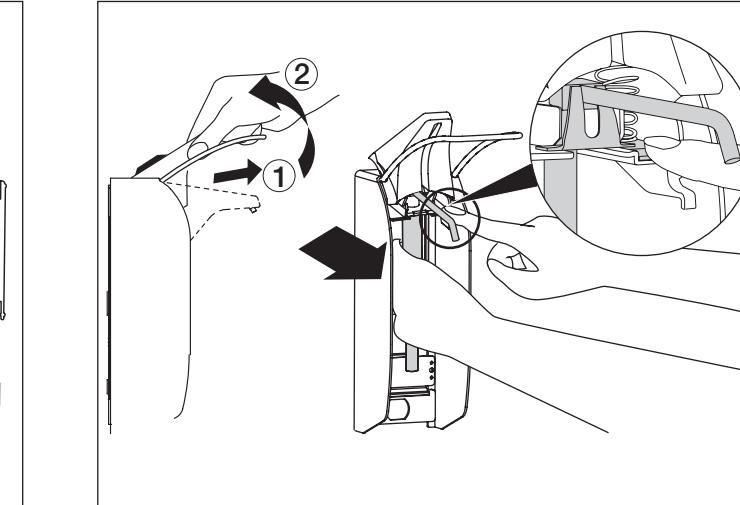
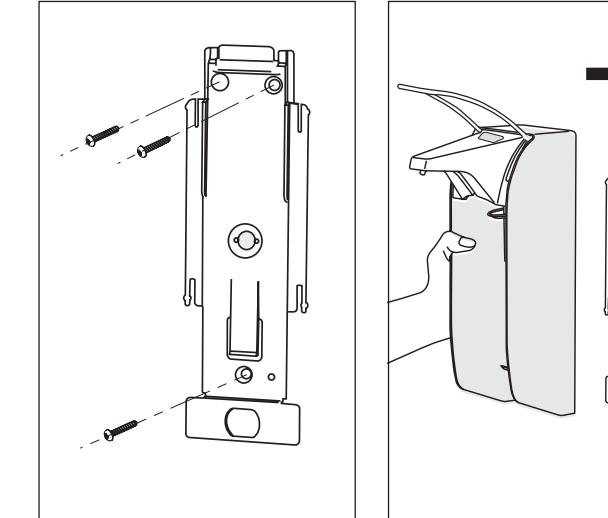
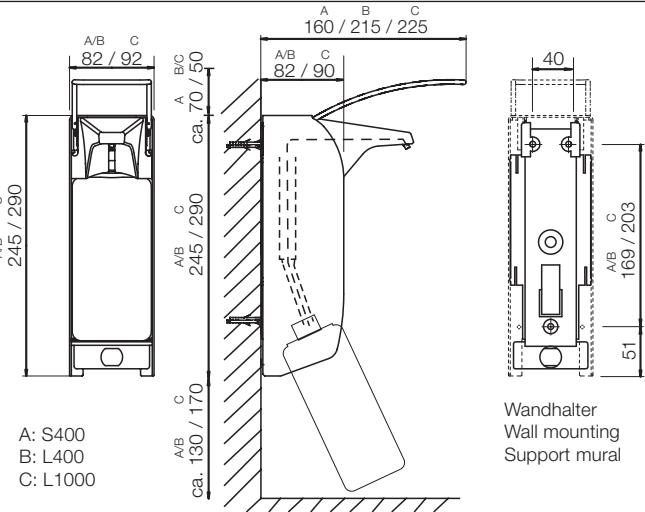
EN 1 Stainless steel operating lever
2 Drip cover
3 Dispenser nozzle
4 Flask

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EN 1. Safety

- Observe the Installation and Operating Instructions.
Trouble-free function of the dispenser can only be ensured where the instructions in these Installation and Operating Instructions are carefully observed.
- The dispenser is only designed for use with liquid soaps (containing no abrasive supplements), fluid lotions or disinfectants.
Using preparations which are unsuitable can lead to reduced function or cause preparations to splash into the eyes.
Preparations containing salts or chloric compounds can cause signs of corrosion and lead to a loss of function.
- Where the dispenser is used in rooms with salty or chlorous air (e.g. in swimming pool or sauna) signs of corrosion can arise on the operating lever.
- Do not dismantle the housing.
- Only use original working parts or spare parts.

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EN 2. Installation instructions

- Set up the appliance above or in the immediate vicinity of the washbasin in order to avoid dispenser preparations from dripping from the hands onto the floor.
- A suitable height for mounting is approximately 130 - 135 cm measured from the floor to the top surface of the dispenser
- Take into account the space necessary to operate the dispenser:
 - Above the dispenser: ca. 150 - 170 mm
 - Below the dispenser: ca. 130 mm (for S400 and L400) or ca. 170 mm (for the L1000 model)

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EN 3. Mounting the dispenser

- i** When replacing an older S400, L400 or L1000 Universal Dispenser with a newer model the existing wall mounting can be used.

- Lift the flask for the preparation being used upwards slightly and remove from the dispenser assembly.
- Press the locking catch of the wall mounting and slide the dispenser assembly upwards.
- Mount the wall mounting plate to the wall using suitable mounting materials.
- Slide the dispenser from above onto the wall mounting plate until the locking catch engages fully.

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EN 4. Changing the pump

Disassemble the pump

- Pull the drip cover lightly towards the front (1) and then tilt upwards (2).
- Place one finger onto the pump block and hold while sliding the pump forwards (see inset).

Assemble the pump

- Place pump in position and slide to the back until the pump catch locks.
- Swing the drip cover downwards.

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
EN 5. Insert flask

- i** The flexible seal cap ensures optimum pressure and prevents any evaporation of the alcohol from the disinfectant.
- Pull the flexible seal cap from below over the pump.
 - Place the flask from below into the dispenser unit.

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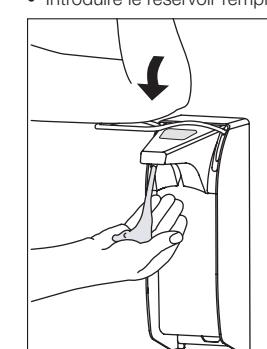
EN 6. Initial use of dispenser

- After insertion of the pump or after inserting the flask with fluid place one hand below the dispenser cover and then operate the lever a number of times until liquid from the dispenser appears.

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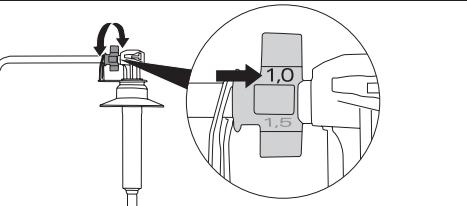EN 7. Setting required dose

The dispenser is set to a maximum dose rate of ca. 1.5 ml per operation (i.e. pressing the lever once). This dose can be reduced to ca. 1.2 ml, 1.0 ml or 0.75 ml. The actual amount depends on the viscosity of the preparation being used in the dispenser.

The actual dose rate can be read off the adjusting wheel when viewing the pump from the right hand side.

- Removing the pump from the dispenser (see section 4. Changing the pump).
- Turn the adjusting wheel on the pump to the required setting, if necessary use a pair of flat nosed pliers.

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EN 8. General care instructions

In its recommendations for hand hygiene in healthcare facilities*, the Commission for Hospital Hygiene and Infection Prevention at the Robert Koch Institute (KRINKO) states that the extent and frequency of checks and the internal cleaning and reprocessing of dispensers should be defined in a facility's internal hygiene plans.

As the manufacturer, we recommend complete reprocessing twice a year. When using hand washing preparations, due to the increased risk of microbial contamination we recommend that the dispenser and the pump are reprocessed every time the unit is refilled. In addition, the operating lever should be wipe-disinfected every day. Similarly, any visible dirt on the outer housing and on any parts of the dispenser that can be accessed without taking it apart as well as any drips forming on the nozzle should be removed via a wipe disinfection.

* Hand hygiene in healthcare facilities. Recommendation of the Commission for Hospital Hygiene and Infection Prevention (KRINKO) at the Robert Koch Institute (RKI), published in the Federal Health Bulletin 2016 · 59:1189–1220

Preparatory measures

Wear protective gloves.

Wear protective goggles.

Use a mask.

Use protective clothing.

- Remove the preparation bottle from the dispenser housing.
- Remove the dosing pump from the dispenser.
- Remove the dispenser from the retaining system or from the rear panel of the dispenser.

Cleaning measures

- Wipe down the riser tube of the dosing pump with a hygienic disposable wipe.
- Clean the dosing pump under running hot water – pay special attention to the nozzle.
- Pumps that have been used to dose hand washing preparations should also be rinsed through with hot water.
- Clean the dispenser housing with water and a hygienic disposable wipe – stubborn residue and dirt should be removed with a brush.
- Allow the dispenser housing and the dosing pump to dry completely.

Disinfection measures

- Once the components have dried completely, wipe-disinfect the dispenser housing, the dosing pump and the rear panel of the dispenser with a suitable surface disinfectant.
- Rinse through the dosing pump with hand disinfectant.

Observe the required contact time.

Sterilisation measures

- The stainless steel dosing pumps can be autoclaved up to a maximum temperature of 134°C, a maximum pressure of 2 bar and a maximum hold time of 20 minutes.

Assembly

- Fit the dispenser on the retaining system or on the rear panel of the dispenser.
- Insert a new dosing pump or one that has been reprocessed separately.
- Insert a new preparation bottle.
- Operate the pumping mechanism to get the preparation flowing.

Automatic reprocessing in the washer-disinfector

- Automatic reprocessing in the washer-disinfector is possible for stainless steel dosing pumps. For automatic reprocessing in the washer-disinfector, standardised and qualified programs are to be used with pH-neutral cleaners and defoaming agents.

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