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CONTENTS

Important Information

1. Notes	12
1.1 CE - Labeling	12
1.2 General notes	12
1.3 Electromagnetic compatibility	12
1.4 General safety notes	12
1.5 Safety notes concerning electric current	12
1.6 Warnings and symbols	13
2. Product information	13
2.1 Correct Usage	13
2.2 Incorrect Usage	13
3. Model overview	14
4. Delivery contents	14
5. Technical Data	14

Installation

6. Transport protection	14
7. Set-up	14
8. Electrical connection	15

Operation

9. Variosuc on/off switch	15
10. Final procedures	15
11. Suction unit cleaning	16
11.1 After every patient	16
11.2 Each day before lunch and after work	16
11.3 Secretion tank	16
12. Filter change	17
12.1 Hose manifold	17

Waste disposal

13. Correct Disposal	17
-----------------------------------	----

Spare parts list

Variosuc Model 0626	18
---------------------------	----

IMPORTANT INFORMATION

1. NOTES

1.1 CE - Labeling

CE-Labeling in line with Medical Product Guidelines for class IIa-products. This product carries the CE-labeling "CE-0124" conforming to the Guidelines of the Committee for Medical Products 93/42/EWG and fulfills all requirements of Appendix I of these Guidelines. This product is a class IIa-product (MPG).

1.2 General notes

- The operating instructions are part of the appliance. They should always be at hand near the unit. Accurate observation of the instructions is a pre-requisite for correct usage and handling of the appliance.
- The safety of operators and trouble-free operation of the appliance can only be guaranteed when original parts are used. Furthermore, only parts as listed in the technical documentation or parts explicitly approved by Dürr Dental may be used. Where parts or products from other manufacturers are used, Dürr Dental can guarantee neither trouble-free operation nor functional safety.
- No claims for compensation of any kind will be accepted, where damage results from the use of parts or products from other manufacturers.
- Dürr Dental accepts responsibility for the safety, reliability and function only when installation, resetting, alterations, additions and repairs have been carried out either by Dürr Dental or by a responsible party authorized by Dürr Dental, and where the appliance has only been used in accordance with the operating instructions.
- The operating instructions correspond with the appliance and the technical safety standards. Copyright protection is reserved for the listed switches, techniques, names, software programs and appliances.
- Copying of all or part of the technical documentation is only possible where written permission of Dürr Dental has been obtained.

1.3 Electromagnetic compatibility

The requirements of EN 60601-1-2 "for electromagnetic compatibility - electrical medical appliances" have been met.

1.4 General safety notes

- Dispose of all packing material according to regulations. Take care that it does not fall into the hands of children.
- This product is a medical technical appliance and may only be used by those persons whose training and/or experience guarantees correct usage.
- Before each use of the appliance the operator should check both the safety and that it is in fully working order.
- The operator must be fully aware of how to use the appliance correctly.
- This product is not suitable for use in rooms where there is the danger of explosion. Areas where there is an explosion hazard are those where flammable anesthetic, skin cleaning agents, oxygen or skin disinfectants are present. Furthermore, this product is not suitable for all environments where there is the possibility of fire breaking out.

1.5 Safety notes concerning electric current

- The Variosuc may only be connected to a standard approved earthed electrical socket.
- Before connecting the appliance to the mains, check that the frequency and voltages given for the appliance match those of the local power supply.
- Extension cables must not be used for electrical connection to the appliance.
- Before commissioning the appliance all connections must be checked for possible damage. Damaged connections, plugs and sockets should be replaced immediately.
- Never touch patients and open sockets of the appliance simultaneously.

1.6 Warnings and symbols

The operating instructions and packaging contain the following labeling and symbols for especially important information.

Restrictions and regulations concerning the prevention of injury or damage.

Warning concerning dangerous electrical voltage.

Special instructions concerning economic use of the appliance or other notes.

Ground-free part type B

CE-Labeling with Notified Body Number

CE- Labeling without Notified Body Number

Use once only

Saliva cannula

Wear impermeable gloves

Aspirate cold water

Main switch ON-OFF

Observe operating instructions

2. PRODUCT INFORMATION

2.1 Correct Usage

The Variosuc has been designed to aspirate spray mist, secretions and saliva. Appliances may only be used together or may only be connected together after it has been checked that the safety of patients, operators or to the environment will not be reduced through such connection.

In cases where the complete safety of the appliance on connection cannot be guaranteed through the appliance data, then the operator must check that the safety of patients, operators or to the environment will not be reduced through such connection, e.g. by contacting the manufacturers or checking with qualified personnel. The standards IEC 601-1 (EN 60601-1) and IEC 601-1-1 (EN 60601-1-1) must be adhered to.

2.2 Incorrect Usage

Incorrect usage is all usage above and beyond that explicitly laid down in the operating instructions. The manufacturer accepts no liability for damage resulting from other use. All risk is carried by the operator.

3. MODEL OVERVIEW

Model 0626

Variosuc with suction unit 0740, on rollers, secretion tank, economy-rack

4. DELIVERY CONTENTS

Variosuc	0626
Fuse Set	9000-115-25
Orotol-Plus Test-Set	XX02-92-009
Cleaning Set Orotol	0780-300-00
Cannula-Set	0700-009-00
One-way Filter	0725-110-00
Operating and Installation Instructions Variosuc 0626	9000-605-55/01
Directions for Use Disinfection & Cleaning of Suction System	9000-605-10
Installation Instructions Economy-Rack	9000-605-57

5. TECHNICAL DATA

Typ		0626
Voltage	V	230
Frequency	Hz	50
Current	A	2,7
Initial Current	A	7
Power	kW	0,6
Duty cycle	%	100
Vacuum	mbar (p _e)	-90
Safety Class		I
Protection Class		IP 21
Noise Level according to EN 21680-1	dB(A)	58
Size, HxBxT	(cm)	87x34x46
Weight	(Kg)	29
Temperature range		
Operating appliance		+ 10 bis+40°C
Storage and transport		- 25 bis+70°C
Humidity		
Operating appliance		max. 80%
Storage and transport		max. 95%

INSTALLATION

6. TRANSPORT PROTECTION

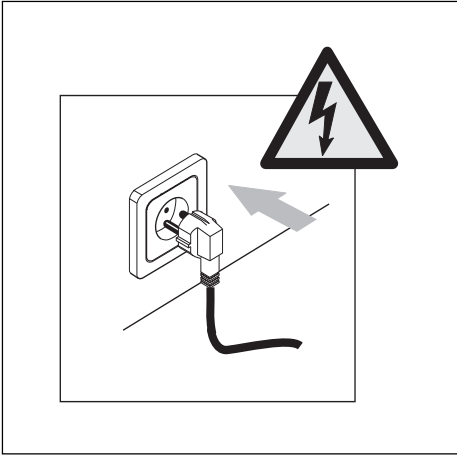
The motor plate, which is fitted on rubber buffers, is held firm during transport by four screws.

This transport protection must be loosened before operating the Variosuc. Hereby:

- Remove the gray cover from the motor housing.
- Loosen the four screws placed in each corner of the motor plate until the motor plate can move easily over the four rubber buffers.
- Replace the gray cover on the motor housing.


7. SET-UP

- Before putting into operation check that the power supply corresponds with that on the appliance identification plate.
- In winter the appliance should only be put into operation after transport after it has warmed to room temperature (build-up of condensation can lead to short circuiting)
- The Variosuc should be placed as near to the treatment area as possible.

8. ELECTRICAL CONNECTION

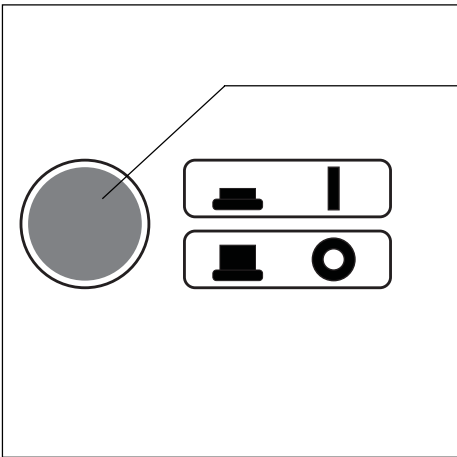The mobile Variosuc 0626 must be connected to an earthed socket.

OPERATION

1 9. VARIOSUC ON/OFF SWITCH

Before working turn the Variosuc on at the main switch (1). The lamp at the switch is on when the appliance is ready for operation.

Secretion from the patient's mouth is aspirated via the suction hoses and carried to the secretion tank. This should be emptied at least once a day.

10. FINAL PROCEDURES

After installation check all hoses for possible leakage. The Variosuc requires no connection to waste water outlet.

Exhaust air is passed through a muffler back into the appliance

11. SUCTION UNIT CLEANING

Please observe the hygiene steps for dental surgeries.

11.1 After every patient

For both hygiene and functional reasons one glass of **cold** water should be sucked through by both the **wide** and the **narrow** suction hose - even if only the narrow hose was used for treatment.

Suction with the **wide** suction hose causes a large amount of air to be sucked in and the cleaning effect that results is considerably improved.

11.2 Each day before lunch and after work

The suction unit should be cleaned and disinfected by aspiration of a suitable cleaning and disinfecting agent, e.g. OROTOL Plus.

Rinse the suction unit thoroughly with water before disinfecting. Use no foam-producing agents e.g. household or instrumentation cleaner!

Further information can be found in the documentation **“Disinfection and Cleaning of the Suction System”**
Order-No. 9000-605-10

11.3 Secretion tank

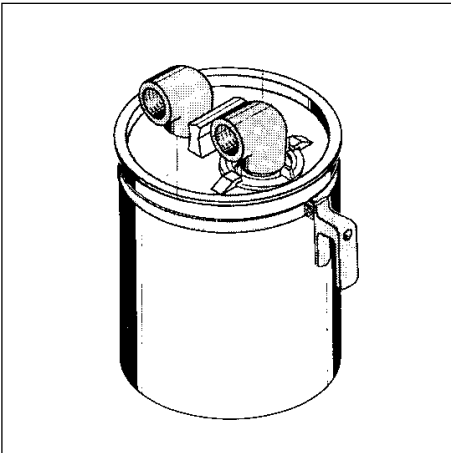
Empty, clean and disinfect the secretion tank every day, and at the latest when the float switch in tank lid is activated and the power disconnected.

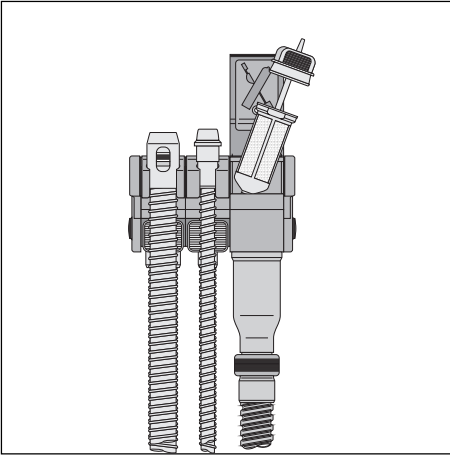
Wear impermeable gloves when emptying and cleaning the tank.

If amalgam is also sucked up during operation, this should be disposed of according to the relevant regulations.

12. FILTER CHANGE

Wear impermeable gloves.

12.1 Hose manifold

The one-way filter in the hose manifold should be changed at least 1x week

WASTE DISPOSAL

13. CORRECT DISPOSAL

Heavy metal particles and amalgam dust from fillings are carried through the hose.

In order to reduce the heavy metal content in the waste water resulting from dental treatment appliances this waste water should be disposed of according to all the relevant national regulations.

Amalgam-coated parts such as grilles, filters and hoses etc. should be disposed of according to the relevant regulations.

The Variosuc 0626 itself can be disposed of normally.

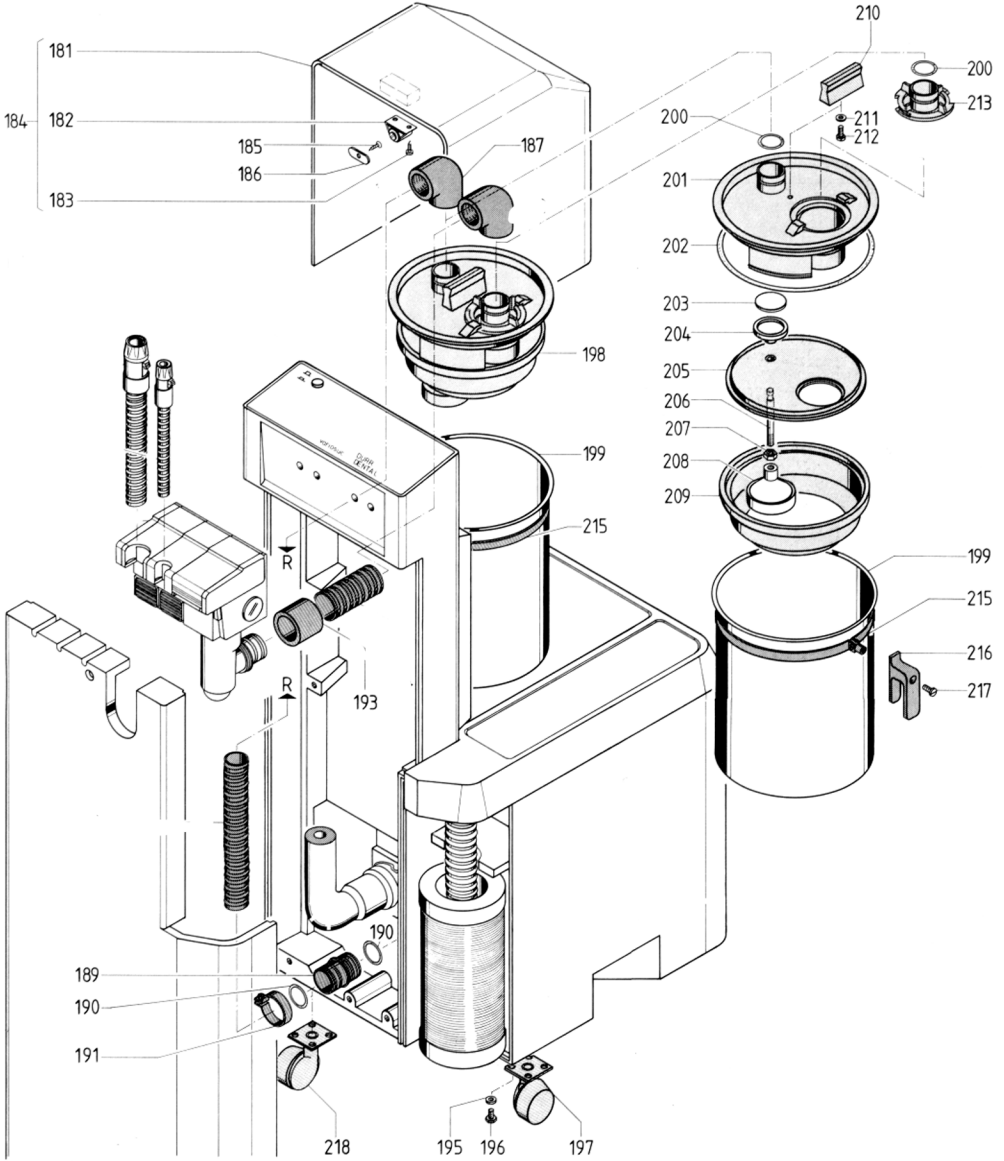
The integrated controller, electronic boards and components should be disposed of as electronic waste.

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VARIOSUC

MODEL 0626

All repairs, above and beyond normal maintenance, may only be carried out by qualified personnel or by our after-service staff.

The following spare parts list does not imply permission to repair.

When ordering spare parts always give the appliance model and serial number. This information can be found on the appliance name plate.

181 Hood	0626-020-01
182 Magnetic latch	9000-429-07
183 Sheet metal screw	9000-460-62
184 Cover complete	0626-020-00
185 Flat head screw	B2.9x9.5 Niro DIN 7972
186 Plate	9000-429-08
187-191 Hose pieces	0626-030-00
187 Elbow joint	0785A-010-03
189 Hose support	0630-041-02
190 O-Ring	9000-401-07
191 Hose clamp	9000-160-19
192-194 Secretion hose	0626-010-00
193 Hose support	7066-000-02
195 Washer	B5.3VZ DIN 125
196 Screw	M5x10 VZ DIN 933
197 Caster, left	9000-408-06
198 Tank lid	0710-100-00
199 Tank with sealing ring	0659-020-00
200 O-Ring	9000-401-07
201 Lid insert	0710-100-02
202 O-Ring	9000-401-08
203 Seal	0710-112-02
204 Valve plate	0710-112-01
205 Lid floor	0710-110-01
206 Float spindle	0710-110-02
207 Washer (plastic)	M6 DIN 934
208 Float	0710-111-00
209 Lid base	0710-100-03
210 Lid handle	9000-154-09
211 Washer	B4.3 VZ DIN125
212 Machine screw	M4x10 VZ DIN 84
213 Filter cover	0710-101-00
215 Sealing ring	0711-060-00
216 Tank handle	9000-154-08
217 Machine screw	144x10 VZ DIN 84
218 Caster, right	9000-408-05

Dürr Dental GmbH u. Co.KG

Höpfigheimer Straße 17 · 74321 Bietigheim-Bissingen/Germany
Tel. (07142) 705-0 · Fax (07142) 6 13 65 · <http://www.duerr.de>

10/99 deutsch
10/99 engl.