

# Product information

## FD multi wipes/FD multi wipes compact

### Nonwoven wipes



#### FD multi wipes/FD multi wipes compact at a glance

- Premium-quality, tear-resistant, lint-free, dry wipes in a refillable dispenser.
- General purpose wipes for use with working solutions of FD 300 and FD 312 and with ready-to-use solutions of FD 322, FD 333, FD 333 forte or FD 366 sensitive.
- Finely structured, nonwoven material with excellent wetting properties that won't scratch surfaces.
- Extra-large wipes sized 200 x 300 mm or 290 x 300 mm.
- Proven disinfecting action and stable solutions.
- Effective from the first day to the last – shelf life of 28 days to 3 months.
- Practical, resealable dispenser for refilling.
- Highly economical – enormously time-saving: the disinfecting wipes only need to be soaked once, reducing disinfectant consumption. Sufficient for 180 or 60 disinfection procedures, respectively.

#### Properties

FD multi wipes/FD multi wipes compact from Dürr System Hygiene are premium-quality, tear-resistant wipes. FD multi wipes/FD multi wipes compact can be used with a wide range of surface disinfectants. Dürr Dental recommends working solutions of the FD 300 or FD 312 concentrates and ready-to-use solutions of FD 322, FD 333, FD 333 forte or FD 366 sensitive. FD multi wipes/FD multi wipes compact provide for excellent distribution of the disinfectant solution, ensuring the complete wetting of the surfaces to be disinfected. FD multi wipes/FD multi wipes compact are sized and made of materials that can be easily folded to reach surfaces that are small or difficult to access. Using FD multi wipes/FD multi wipes compact reduces preparation times for disinfection procedures and disinfectant consumption. The dispenser protects the soaked FD multi wipes/FD multi wipes compact against contamination and evaporation.

#### Product composition

FD multi wipes are sized 200 x 300 mm. FD multi wipes compact are sized 290 x 300 mm.

#### Directions for use

Use FD multi wipes/FD multi wipes compact to clean and disinfect surfaces on treatment units, work surfaces, walls, medical equipment, and objects of all kinds. Produce a 1-3% working solution of FD 300 or a 1-2% working solution of FD 312 based on the desired exposure time. FD 322, FD 333, FD 333 forte, and FD 366 sensitive are ready-to-use solutions that are applied undiluted. Soak the wipes evenly with 2.5 L solution in the FD multi wipes dispenser or with 1.4 L solution in the FD multi wipes compact dispenser, thread the top wipe, seal the dispenser tightly, and wait 30 to 60 minutes before using the first wipe. Enter the product name, concentration, batch number, production date, expiry date, and name of the responsible person on the label enclosed with each roll of wipes and adhere the label to the resealable lid on the dispenser. Remove a wipe and use it to completely wet the surface to be disinfected. Allow the exposure time of the disinfectant in use to elapse as described in the instructions for use. Reseal the dispenser tightly after each use. Before refilling or replacing the product, empty the

# Product information

## FD multi wipes/FD multi wipes compact

### Nonwoven wipes

dispenser of any remaining wipes and liquid, remove and dispose of the label from the lid, and process the dispenser. Manual processing: Manually scrub and remove any deposits or residues. Thoroughly rinse the inside and outside of the dispenser, including the lid, with hot tap water (> 55°C - Caution: risk of scalding!) and dry with a paper towel. Then use a paper towel soaked with a fast-acting, alcohol-based disinfectant (e.g. FD 322 or FD 333) to completely wet and thoroughly disinfect the dispenser and lid. (Allow corresponding exposure time to elapse!) Make sure to dry the dispenser and lid completely. Alternative automated processing: Process and dry the dispenser and lid in a chemothermal cleaning and disinfection system at up to 93°C. Place a new roll of wipes in the clean and dry dispenser and prepare as described above: soak the wipes, thread the top wipe, seal the dispenser tightly, adhere a new label to the lid, and wait 30 to 60 minutes before using the first wipe. Do not process contaminated dispensers, dispose of them. Freely protruding wipes may be contaminated and must be discarded. Do not place removed wipes back into the dispenser. Do not use wipes that have dried out. Replace defective dispensers.

#### Environmental impact

The dispenser is made of polypropylene and can thus be recycled or incinerated. Dispose of used wipes with the domestic waste. Do not flush.

#### Shelf life

Use soaked wipes within 28 days to 3 months. For more detailed information, see the summary of product characteristics for the products on our website.

#### Package sizes

##### FD multi wipes:

Dispenser with 1 roll of 180 wipes.

Refill pack with 4 rolls of 180 wipes each in the box.

##### FD multi wipes compact:

Dispenser with 1 roll of 60 wipes.

Refill pack with 8 rolls of 60 wipes each in the box.



#### Sales

Dürr Dental SE  
Höpfigheimer Str. 17  
74321 Bietigheim-Bissingen  
Germany  
[www.duerrdental.com](http://www.duerrdental.com)  
[info@duerrdental.com](mailto:info@duerrdental.com)

#### Manufacturer

orochemie GmbH + Co. KG  
Max-Planck-Straße 27  
70806 Kornwestheim  
Germany  
[www.orochemie.de](http://www.orochemie.de)  
[info@orochemie.de](mailto:info@orochemie.de)