

# Product information

## FD 360 cleaning and care of vinyl upholstery



### FD 360 features in brief

- Cleaning fluid for cleaning and caring of vinyl upholstery.
- Intensive cleaning effect – removes stains of all kind, spots and discoloration.
- Excellent material compatibility – suitable for all covers made of artificial leather.
- Environmentally friendly propellant-free spray bottle.
- Set with one 500-ml spray bottle and 10 special sponges.

### Characteristics

FD 360 from Dürr System-Hygiene cleans and cares for all vinyl upholstery covers of treatment chairs and is highly effective in maintaining the value of dental treatment chairs and other artificial leather covers. You can apply FD 360 with the special sponge supplied in the set to rapidly and thoroughly remove even stubborn grime, stains, and discoloration, caused for example by non-fast dyestuffs in clothing. FD 360 cleans and cares intensively without leaving any residues and provides a silky gloss finish.

### Chemical composition

FD 360 is based on a mixture of special surfactants, silicone compounds and care components on the basis of avocado oil.

### Directions for use

Use FD 360 cleaning fluid undiluted. Spray FD 360 on the surfaces to be cleaned and immediately wipe with a cloth or, for intensive cleaning, spread FD 360 by means of the special sponge, which produces an even more intensive and deeper cleaning effect. FD 360 in combination with the special sponge removes the most stubborn stains and discoloration from the covers. Subsequently, wipe off the excess FD 360 with a dry cloth. Use FD 360 daily, if required.

### Environmental behaviour

FD 360 is ecologically safe. Application by means of a propellant-free patented spray bottle. The surfactants meet the requirements for biodegradability. The packaging can be exploited both materially and thermally. For recycling, rinse bottle with water.

### Physical properties

Appearance: white opaque solution

Density:  $D = 0.99 \pm 0.05 \text{ g/cm}^3$  (20°C)

pH  $3.6 \pm 0.5$

### Shelf life

3 years

# Product information

## FD 360 cleaning and care of vinyl upholstery

### Packaging unit

Set with one 500-ml spray bottle + 10 special sponges in a carton

### Storage

Store product at 5°C to room temperature max.

### Caution

FD 360 can be used without any problems on all artificial leather surfaces. The material tolerability of other leather surfaces should be checked on a concealed surface. The special sponges must not be used on genuine leather.

### Hazard Statement

FD 360 is classified according to the CLP Regulation and is not required to be labelled.