

Sterilization, Hygiene and Maintenance

#whdentalwerk
f @ in ▶
video.wh.com

Now at
your dealer
or wh.com



Interrupting the chain of infection.

Good hygiene management is an essential part of everyday work in dental practices, because it is all about the safety of your patients, your staff and yourself.

Sterilization, Hygiene and Maintenance

COMPANY	4–5
Product overview for Teon, Teon +, Assistina Twin, Assistina One	8–9
Product overview for Lisa, Lara XL, Lara, Lina	10–11
CLEANING AND DISINFECTION	
Teon/Teon +	12–13
REPROCESSING	
Assistina Twin	14–15
Assistina One	16–17
STERILIZATION	
Lisa, Lara XL, Lara, Lina, Lyla, Lexa	18–29
Accessories	30–31
SEALING	
Seal ² /SealVal ²	32–33
SECURE PACKAGING	
EliReel	34–35
WATER TREATMENT	
Multidem, Osmo, Water Distiller, Prim	36–41
CLEANING AND DISINFECTION AGENTS	
BePro	42–43
TECHNICAL DATA AND ADAPTORS	
Technical data	44–49
W&H adaptor set	50
System overview adaptors	51

People have Priority

W&H at a glance

- › Headquarters in Austria
- › 1,300 employees worldwide
- › Established in 1890 in Berlin by Jean Weber and Hugo Hampel
- › Production facilities in Austria, Italy and Sweden
- › Represented in 130 countries
- › Export share over 95%

More information at
wh.com



W&H Dentalwerk
Bürmoos (A)



W&H Sterilization
Brusaporto (I)



The stakes are high.
We are working with
people.



Healthy development occurs when action is taken with foresight. This applies for sterilization, hygiene and maintenance just as it does for W&H: after all, as a family-owned company, we have been acting with the appropriate far-sightedness for generations.

The secret to our success: W&H has always focussed on people – regardless of whether they are partners, customers, patients or employees. It would only be natural for you now to ask critically: Is that really put into practice? Our suggestion: put us to the test!

W&H AIMS
Safety first!

Hygiene is extremely important in the dental practice. W&H AIMS is here to help: with a wide product portfolio, fundamental knowledge and all relevant information at a glance.

For your safety and the safety of your patients. [aims.wh.com](https://www.aims.wh.com)

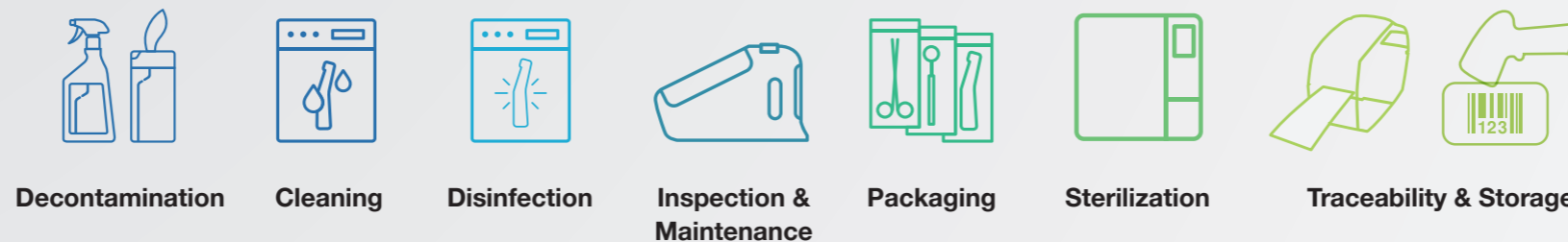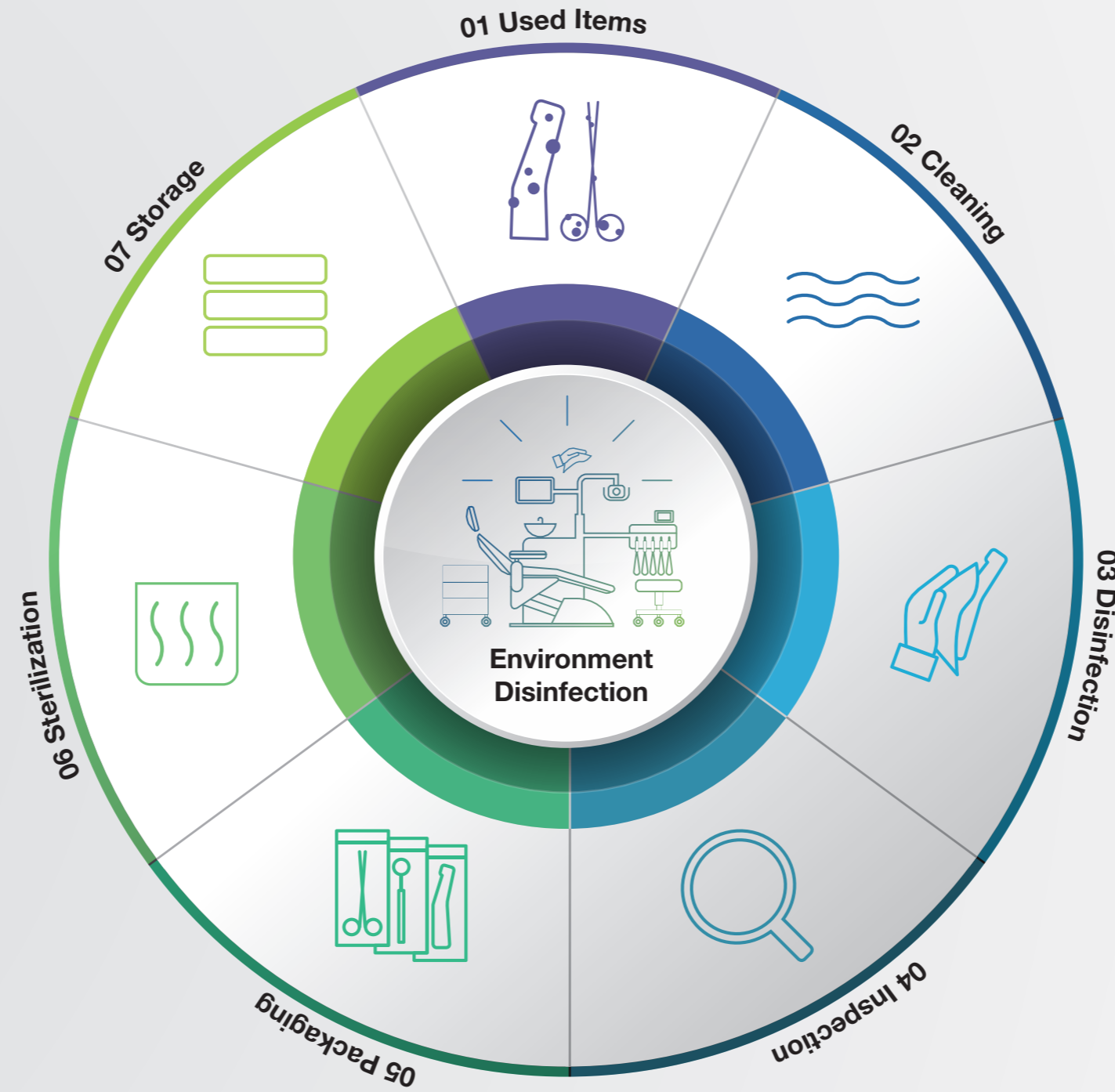

All the **advantages**
at a glance



Dental assistant

“The Teon fits in perfectly as part of our entire reprocessing process and simplifies our workflow significantly.”



Teon, Teon +

- › High chamber capacity and high cleaning performance
- › User-friendly display
- › Gentle on material thanks to dedicated programs
- › Large selection of accessories
- › Cleaning of the hollow body
- › Traceability and connectivity through Ethernet module

Teon + offers in addition:

- › High drying quality for sensitive instruments
- › Active air-drying with HEPA filter

[More information from page 12](#)



Dr Christian Müller

Dentist with his own practice

“Now I can be certain that my instruments can be reprocessed between treatments without my patients and I having to wait. A real bonus for my dental practice.”



Assistina Twin with Quick Connect by W&H

- › Easiest operation
- › Instrument reprocessing in just 10 seconds
- › Non-stop assembly through two process chambers that can be used alternately
- › Assistina Twin Care Set sufficient for reprocessing up to 2,800 instruments
- › Active suction of aerosols with integrated HEPA filter
- › Wide range of adaptors

[More information from page 14](#)



Assistina One

- › Instrument reprocessing in just 18 seconds without electrical supply
- › New quick adaptors to speed daily activities
- › Assistina One Care Set sufficient for reprocessing up to 2,800 instruments
- › Integrated HEPA filter to meet high hygiene standards
- › Blowing through and drying with compressed air

[More information from page 16](#)



All the **advantages** at a glance



Dr Klaus Kohlpaintner
Dentist with his own dental practice

“Lisa is an efficient sterilizer and very easy to use. It makes our daily work much easier!”



Dr Jakob Svoboda
Dentist with his own dental practice

“For us, the Lina is the ideal basic model for economical instrument reprocessing.”



Lisa

- › Easy to use with crystal-clear colour touch screen allowing intuitive navigation via the intelligent menu structure
- › EliSense technology for optimisation of daily workflow via LED indicators and touch screen
- › EliTrace technology for traceability of the sterilization process right down to the individual instrument and user
- › High level of additional security with Wi-Fi and W&H Steri App for controlling and real-time remote monitoring
- › High efficiency with patented Eco Dry + technology for reduction of cycle time
- › High level of ergonomics thanks to newly developed design
- › Programmable cycle start
- › Automatic water filling valve

[More information from page 18](#)



Lara XL

- › High efficiency with patented Eco Dry + technology
- › High capacity with 28 litre chamber
- › Individual upgrades with Activation Code system
- › Intelligent menu structure with high operating comfort
- › Additional security with Wi-Fi and W&H Steri App for controlling and real-time remote monitoring
- › Easy to use thanks to colour touch screen display and intuitive navigation

[More information from page 22](#)



Lara

- › Highest-performing sterilizer in its segment
- › Easy to use thanks to colour touch screen display for intuitive navigation through the intelligent menu structure
- › Individual increase of efficiency through upgrades with Activation Code system
- › High level of additional security with Wi-Fi and W&H Steri App for controlling and real-time remote monitoring
- › Paper-free documentation with high-capacity USB stick for automatic recording of all cycles
- › Ergonomic design

[More information from page 22](#)



Lina

- › The ECO B cycle reduces the cycle time by half
- › Colour touch screen display for simple navigation to all sterilization options
- › Automatic digital cycle recording on USB stick
- › Individual increase of efficiency through upgrades with Activation Code system
- › High level of additional security with Wi-Fi and W&H Steri App for controlling and real-time remote monitoring
- › Excellent value for money

[More information from page 24](#)



High disinfection standard

Top performer for high cleaning standards.

W&H's Teon and Teon + thermal washer disinfectors provide high quality benefits for efficient and effective cleaning and disinfection of your instruments.



Teon, Teon +



Smooth surface and modern design

The thermal washer disinfectors are distinguished by their high-quality performance and thus ensure a successful, holistic cleaning process for dental instruments. Thanks to their modern, elegant design, they can be perfectly integrated into your hygiene room.



Safety

The integrated spray arm monitoring enables excellent reprocessing results. The speed variable pump increases/decreases the pressure according to the step in the reprocessing process. The filter system removes dirt particles to prevent damage occurring to your instruments.



Convenience and ergonomics

The salt container integrated into the door ensures easy and convenient refilling. Thanks to the ergonomics concept, no equipment needs to be removed.



Rinsing chamber for optimal hygiene

The rinsing chamber enables higher throughput with excellent hygiene and purity. In addition, the optimal size allows a higher capacity per load.



Ease of use

The modern design and intuitive user guidance make it particularly easy to operate in the demanding everyday reprocessing routine. The display is easy to operate and also serves as a door opener.

Lubrication in record time

Simple, fast, efficient.

With its innovative dual-chamber system, record time lubrication and the environmentally friendly Care Set, it impresses all along the line. The Assistina Twin has now been enhanced with an innovative adaptor solution: **Quick Connect by W&H**. This adaptor system offers you even more benefits for your everyday practice.



Assistina Twin with Quick Connect by W&H



Simple

It is the first reprocessing device which is equipped with two process chambers that can be used alternately.



Fast

W&H succeeded in decreasing the maintenance process to a record time of just 10 seconds.



Efficient

Thanks to the smart process monitoring, the Assistina Twin always achieves thorough maintenance with minimum consumption. The Assistina Twin Care Set only needs to be changed after around 2,800 instruments.




Groundbreaking technology

Thorough and comprehensive lubrication of the gear parts (orange) and cleaning of spray channels (green) thanks to innovative oil nebulization technology.



Safe

All aerosols are removed from the process chambers through active suction and an integrated HEPA filter.

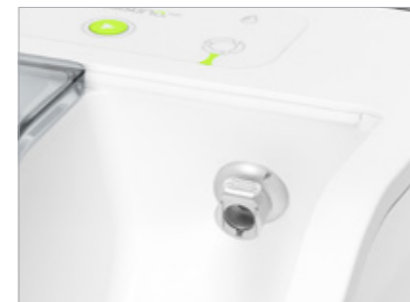

Quick Connect by W&H

The Assistina Twin's innovative Quick Connect by W&H connection simplifies assembly and removal of straight and contra-angle handpieces and turbine handpieces. You can choose the appropriate adaptors for your requirements.

The ONE for a new level of maintenance

Fast, thorough and easy to use

Assistina One speeds up daily operations and facilitates dental handpiece care. With the shortest cycle time in its class and its high-quality care results, the innovative maintenance solution is a top dental device – designed for top dental professionals.



Assistina One
with Quick Connect by W&H



Shortest cycle time

The entire lubrication and spray channel cleaning process only takes 18 seconds, making the Assistina One the fastest maintenance solution in its class. It only works with compressed air; no electric power supply is needed.



Quick Connect System

Flexible switching between handpieces with Quick ISO and Quick RM adaptors. Thanks to its wide range of adaptors, the Assistina One can meet the requirements of all clinics.



Innovative lubrication technology

With the ground-breaking technology of oil nebulisation, dental transmission instruments are lubricated even faster and more thoroughly than ever before.



Spray channel cleaning

By cleaning the spray channels and drying the inner parts with compressed air, it reduces potential blockages caused by limescale deposits and therefore ensures smoother operation, longer lifespan, and less service costs.



Cost-effectiveness process

The system uses exactly the amount of oil required for optimal instrument lubrication. More than 2,800 instruments can be maintained with just one Care Set, saving money, time, and oil spray cans.



Ready-to-use cartridges & Hepa Filter

Convenient cartridge management for refilling oil and cleaning liquids. With the active suction and HEPA filter, aerosols are removed from the process chamber, keeping the working environment clean and safe.

Precision and perfection

Unique in its application.

The Lisa sterilizers offers intelligent and innovative solutions like EliSense and EliTrace for extraordinary user-friendliness as well as traceability down to the individual instrument. The result: convenience, safety and speed at a whole new level.



Lisa



EliSense

EliSense delivers information on the sterilization status, the temperature of the load and to optimise the daily workflow.



EliTrace

With EliTrace, the sterilization process can be traced and documented down to the individual instrument or instrument kit. Without additional software or computer.



Easy to use

The specially developed, crystal-clear colour touch screen guides you intuitively through the intelligent and simple menu structure.



Connected, simple, time-saving

With the W&H Steri App, you can monitor up to four sterilizers in real time using a smartphone or tablet from any place within your practice and all cycles are automatically saved at the same time.



High-performance

The reduced cycle time of 28 minutes for a 2 kg load ensures a longer lifespan of the instruments and reduced energy consumption. The innovative, patented Eco Dry+ technology adapts the drying time to reflect the mass of load.



New design

Its exterior is just as appealing thanks to an ergonomic, space-saving design and elegant, easy-maintenance contours.

Xceed Limits

Lara XL will exceed your expectations. The device convinces with even more capacity, energy-saving and fast type B cycles as well as a wide range of connectivity and advanced traceability options.



Lara XL



Xceeding performance

The reduced cycle time of 40 minutes for a 2 kg load ensures a longer lifespan of the instruments and reduced energy consumption. The innovative, patented Eco Dry + technology adapts the drying time to reflect the mass of load.




Xceeding capacity

With its 28 litre chamber, Lara XL is able to reprocess up to 7 kg of wrapped instruments or 9 kg containers. Thanks to its 6-tray rack, it is extremely flexible.



Xceeding usability

The intelligent menu structure of the Lara XL sterilizer ensures extraordinarily high operating comfort: You can navigate to the different functions quickly and intuitively using the large colour touch display.

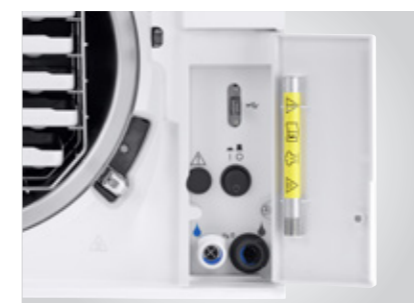

Xceeding connectivity and traceability

The sterilizer offers future-oriented connectivity options. The documentation of sterilization cycles is automatically saved via a high-capacity UBS stick. Wi-Fi and LAN connections offer the possibility to download cycle reports directly to any PC or smart device.



Xceeding upgradeability

With the optional Activation Code EliTrace the sterilization process can be traced and documented down to the individual instrument or kit. Without additional software or computer. The Fast cycle upgrade enables sterilization of unwrapped instruments, including handpieces in just 20 minutes.

As unique as your practice

Easy to expand with customized upgrades.

Sterilize safely, easily and quickly – plus, you can customize the device to suit your needs: the standard version of the Lara sterilizers already offers the fastest sterilization cycle in its class – including standard documentation. An additional, newly developed Activation Code system offers even more speed and extended documentation if you so wish – and ensures you are optimally equipped even if your requirements change.



Lara



Fast, faster, Lara

The standard version for type B cycles already gives you more time for your patients.



Even higher performance

If you need even more speed, you can use an Activation Code to perform a quick upgrade to Eco Dry + and the Fast Cycle for unwrapped instruments.



Built-in data logger

The integrated data logger saves the sterilization cycles on a high-capacity USB stick without additional cables and external devices.



User identification

If you would like to extend the documentation, there is a quick and simple upgrade available offering you the option of user identification and traceability of the approval confirmation.



Barcode label printer

The label printer can be connected to the sterilizer to print barcodes. Combined with the built-in data logger your traceability system is enhanced.

The essentials of sterilization

Simplified work processes.

The Lina sterilizers meet all requirements for your daily practice: simple operation, ergonomic and functional design, appropriate sterilization cycles and upgradeable functions guarantee high efficiency leading to maximum results.



Lina



Ease of use

The touch screen display enables smooth operation. The simplified user interface and menu structure support practices in all daily activities.




ECO B sterilization program

The cycle time can be reduced by half for small loads.



Design and maintenance

The easily removable cover allows effortless access to the water tanks without tools. Thanks to modular feet and compact layout the sterilizer fits in anywhere.

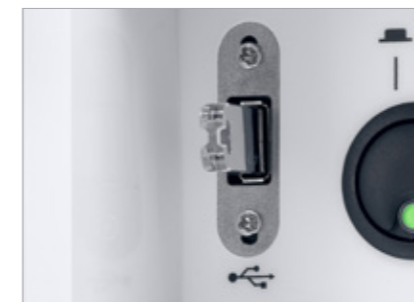

Traceability and upgradeability

The documentation of sterilization cycles is automatically saved via a high-capacity UBS stick. For even more flexibility, use the Activation Code system to activate additional functions to meet your changing practice needs and future requirements.



Barcode label printer

The label printer is a quick and easy tool to link the sterilization cycle with patients files. In combination with the integrated data logger, it improves your traceability system.

Significant for your needs

Discover upgradeability.

The Lyla S type sterilizer offers the best cycle time in its segment and impresses through its upgradeability. The Activation Code system provides Activation Codes that offer a tailored opportunity to enhance additional features that meet your practice needs and comply with future requirements. Effectively, for customising and upgrading your sterilizer.



Lyla



Smart upgrades

The Activation Code system provides several codes that offer a tailored opportunity to enhance additional features that meet your practice needs and comply with future requirements.



Super fast cycles

The different fast cycles available guarantee the possibility to sterilize several instrument types, including hollow bodies, both wrapped and unwrapped.



Secure traceability

With the integrated built-in USB data logger, the optional label printer and the W&H Steri App, Lyla offers a comprehensive traceability system that is customised to your needs.



Simple operation

The Lyla colour touch screen with its simple menu structure has been selected and designed to make your daily work easier, faster and more efficient.

Enhanced traceability thanks to EliTrace

Benefit from a new level of traceability of your sterilizer: From now on, you will not only know exactly which cycles have been carried out at what time and date, but also which instruments were sterilized within this cycle.

The feature is available on Lisa.
More information at wh.com

1

Database Creation

Create your instrument database directly on the display of your sterilizer, on your smart device via W&H Steri App or PC.



2

Labelling and loading

Stick the related pre-printed QR code labels on each sterilization package before loading and make the link cycle-instrument.



3

Load release and status check

After the sterilization cycle, scan directly via W&H Steri App to check the status.



Unique upgradeability thanks to Activation Code system

The Activation Code system is an innovative upgradeability system that offers you a cost-efficient, rapid and customised opportunity to activate additional features to meet your changing practice needs and future requirements.

The Activation Code system is made for Lara XL, Lara, Lina and Lyla.
More information at activation.wh.com



Performance

The "Performance" feature allows even faster cycle times by upgrading to Eco Dry +, which automatically adapts the drying time to the mass of the load.



Traceability

The "Traceability" feature enables user identification and load release confirmation. Identification is secured by a PIN directly on the sterilizer and the W&H Steri App.



EliTrace

With "EliTrace", the sterilization process can be traced and documented down to the individual instrument or kit. Without any additional software or computer.



Fast cycle

The "Fast cycle" feature enables sterilization of unwrapped instruments, including handpieces. For a quick cycle for in-between.



B cycle

The "B cycle" feature upgrades Lyla to B type sterilizer according to EN 13060.



Remote data storage

The "Remote data storage" feature is the ideal addition for the digital cycle documentation. It offers the possibility to save cycles directly on PC.



All in One

The "All in One" feature provides all benefits at once – your sterilizer receives an unattainable price-performance ratio.

Accessories

Compact, practical, good.

W&H offers a wide range of accessories for optimization of the sterilization procedure. Minor extras lead to major results. Work in the dental practice becomes easier and your team saves time.



EliSafe, W&H Steri App, Sprint



EliSafe label printer

The label printer is a quick and convenient tool to link the sterilization cycle with the patient file.



Barcode scanner

The EliSafe package includes a barcode scanner. This enables you to scan labels into the patient file.



W&H Steri App

With the W&H Steri App you are able to monitor up to 4 sterilizers in real time. The W&H Steri App enables you to recall cycle reports using the smart phone camera.



Sprint

To print out cycle protocols on thermal paper with 10-year paper warranty.




Helix test

The Helix test is a test indicator suitable for type B sterilizers.


A perfect sealing process

Safe and secure sterilization package.

It is not easy to keep a product sterile. Only absolutely hermetic and strong protection of the sterile goods from environmental influences guarantees continuing sterility.

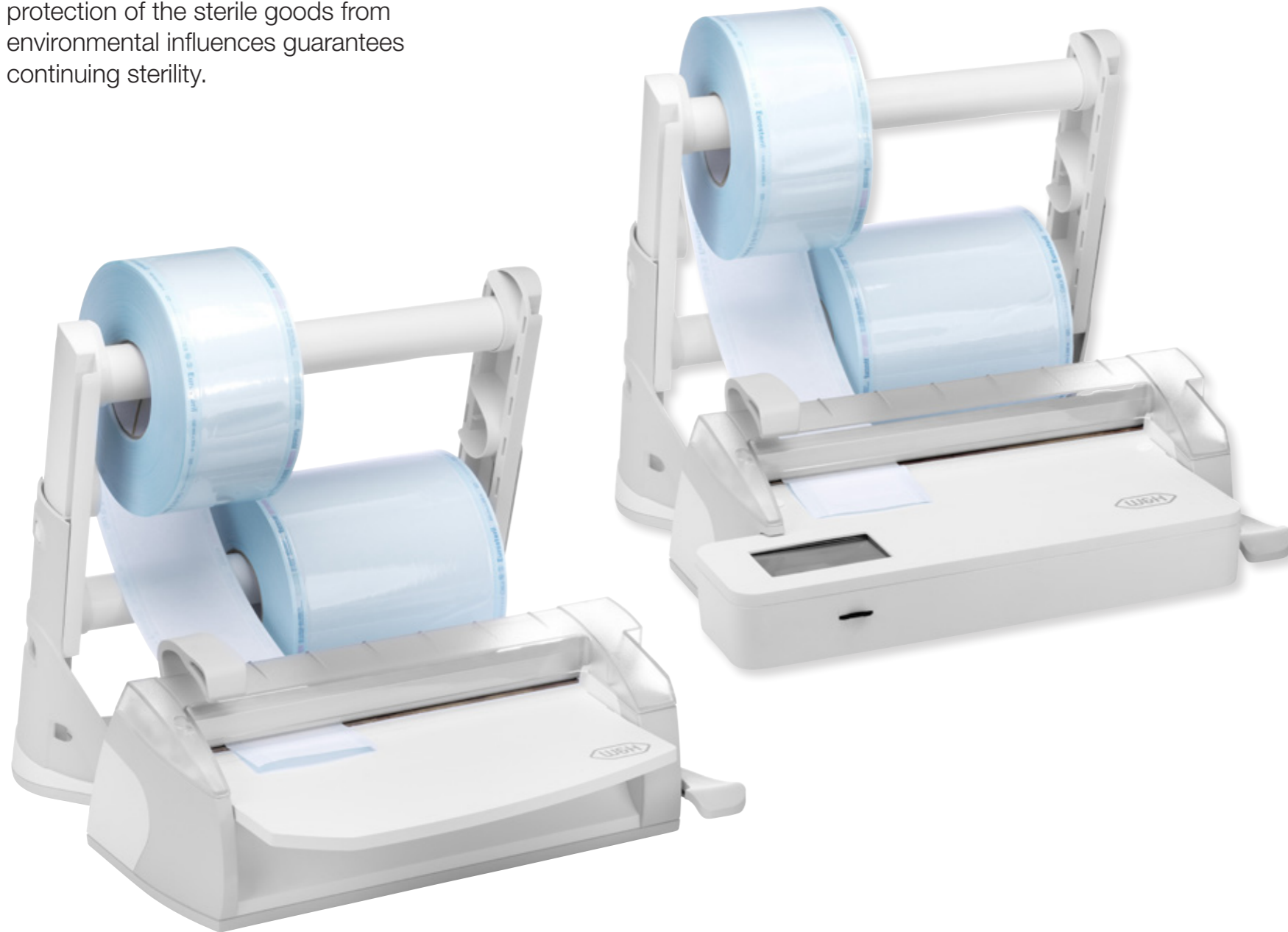

Seal²/SealVal²

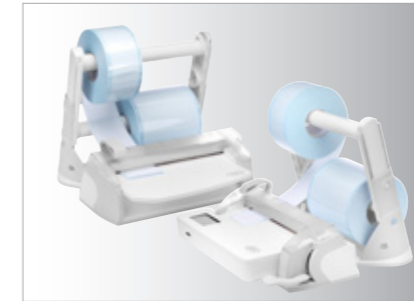

Perfect quality seals

The foil sealer creates a 12 mm wide seal in just two seconds with no risk of burning the sterilization package.



Convenient and user-friendly

Handling is optimized with this transparent sealing station and the integrated work surface.



Flexible working

The patented double-roll holder takes a wide range of widths. The roll holder can be removed from the station and mounted on the wall. This allows flexible working in the smallest spaces.



SealVal² user-friendly touch screen

The touch screen and the intuitive menu structure facilitate the working process.



SealVal² full traceability

The recording of sealing parameters such as pressure, time and temperature ensures full traceability.

Secure packaging for sterile goods

EliReel rolls are the ideal packaging for the safe reprocessing of dental instruments in the sterilizer.



EliReel



Packaging at the highest level

The combination of paper and foil forms a reliable protective barrier against germs. EliReel is particularly resistant and has excellent drying properties.



Ease of use

To simplify daily use and avoid errors, there is only the steam indicator. It quickly and easily shows whether the sterile goods have already been sterilized. EliReel thus meets the highest requirements for sterile goods storage.



Peelable sealing seams

Pouches created with EliReel can be easily opened prior to treatment.

Water treatment systems

Ultrapure water for all systems.

Multidem supplies demineralized water for steam generation and can be used for many autoclaves and reprocessing devices.



Multidem



Compatible

Multidem is suitable for use with steam sterilizers and reprocessing devices. Up to 2 sterilisers can be connected simultaneously.



User-friendly

Cartridges can be replaced fast without tools. The cartridge can be positioned without a fixing.



Practical

Multidem is also equipped with a spray gun and is ideal for rinsing instruments prior to sterilization.



Economical

Multidem does not require an external power supply. It has an optimized ion exchange resin.



Safe

The purified water complies with the recommendations and guidelines for the correct reprocessing of instruments.

Water treatment systems

High-quality demineralized water.

Osmo is the best water treatment device to supply several sterilizers with high-quality demineralised water at the same time.



Osmo



Compatible

Osmo fills and drains up to 6 sterilizers automatically and simultaneously.



User-friendly

The screw-unscrew system makes cartridge replacement easier than ever. The cartridges can be replaced without any tools.



Practical

Osmo has a spray gun for manual filling of other devices or for rinsing instruments prior to sterilization.



Economical

The water storage tank has a capacity of seven litres. This makes Osmo not only the cost-effective solution for regions with poor water quality, but also for practices with a high water demand.



Safe

The purified water complies with the recommendations and guidelines for the correct reprocessing of instruments.

Water treatment systems

Distilled water of exceptional purity.

The Water Distiller enables economical production of distilled water for sterilization and produces pure water from tap water.



Water Distiller



User-friendly

Automatic heater cut-out, compact, silent and very simple to use. The Water Distiller stands out for its ease of use.



Practical

A water receiver is supplied together with the distiller.



Economical

The device supplies 4 litres of distilled water in less than 5 hours. With this volume, up to 8 sterilization cycles can be carried out in our sterilizers.



Prim

Prim checks water quality by measuring the conductivity of the water.

BePro for perfect cleaning and disinfection

BePro protects people from infections and materials from contamination. Depending on the work or treatment step, surfaces, instruments, dental units or hands must be cleaned and disinfected in a hygienically safe manner in everyday practice. The W&H BePro hygiene portfolio covers the entire hygiene product needs of dental practices.

BePro – particularly effective, efficient & compatible

Effectiveness

- › Broad pathogenic spectrum: bactericidal, mycobactericidal, tuberculocidal, fungicidal, virucidal
- › High cleaning and disinfecting efficacy
- › Tested and EN compliant

Efficiency

- › Extremely short exposure times
- › Convenient all-in-one products (cleaning and disinfection)
- › Low dilution rate for concentrates

Compatible

- › Excellent tested material compatibility even on sensitive surfaces (incl. artificial leather and acrylic glass)
- › Gentle to the skin and dermatologically tested
- › Refillable packaging



BePro – Suction Systems

For the cleaning, disinfection and maintenance of dental unit's suction system and spittoon bowl.



BePro – Hand Care

For the cleaning and maintenance of hand and skin health.



BePro – Surfaces

For the cleaning and disinfection of

- › non-invasive medical devices e.g. dental units incl. upholstery, exterior parts of handpieces, patient environment.
- › large surfaces e.g. reception desks, waiting rooms, floors, handles, doors.



BePro – Instruments

For the decontamination of dental instruments e.g. scalers, probes, scissors, pliers, burs, files.



BePro – Automated Reprocessing

For thermal washer disinfectors or washers like the W&H Teon or Teon +.

Teon, Teon +

CE

Type:	Teon	Teon +
Power supply:	400 V AC; 50–60 Hz/230 V AC; 50–60 Hz	
Current:	16 A	
Overall dimension (w × h × d):	600 × 835 × 600 mm	
Space required for installation (w × h × d):	598 × 835 × 1,200 mm (in total with opened door)	
Weight (empty):	74 kg	78 kg
Smart Dry:	•	–
Dry Plus:	–	•
HEPA filter:	–	•
Cold water connection:	•	•
Hot water connection:	–	•
Demineralized water connection:	•	•
Auto-close – automatic door lock:	•	•
Number of integrated dispenser pumps for liquid products:	2	1
Integrated dispenser module for rinse aid:	•	•
Salt tank – located in the front door:	•	•
Number of programs:	5	6
Number of customizable programs:	2	2
Cycle time*:	49'	74' including active air-drying
* time for Vario TD Dental program		

Assistina Twin

CE

Trade name: Type:	Assistina Twin MB-302
Process:	Cleaning of spray channels, lubrication
Suitable for:	Straight and contra-angle handpieces, turbine handpieces, air scalers, air motors
Supply voltage:	100–240 V AC
Maximum power consumption:	18 VA
Air consumption:	approx. 40 NI/min
Operating pressure:	5–10 bar (controlled via an integrated, automatic pressure controller)
Overall dimension (w × h × d):	297 × 190 × 435 mm
Weight:	3.5 kg
Capacity:	200 ml W&H Activefluid cleaning solution, 200 ml W&H Service Oil F1

Assistina One

CE

Trade name: Type:	Assistina One MB-301
Process:	Cleaning the spray channels, lubrication
Suitable for:	Straight and contra-angle handpieces, turbine handpieces, air scalers, air motors
Air consumption:	approx. 50 NI/min.
Operating pressure:	5–10 bar (controlled via an integrated, automatic pressure controller)
Dimensions (w × d × h):	166 × 383 × 243 mm
Weight:	2.2 kg
Capacity:	200 ml W&H Activefluid cleaning solution, 200 ml W&H Service Oil F1

Lisa

CE₀₀₁ CE₀₀₇

Trade name: Type:	Lisa 17 VA131-17	Lisa 22 VA131-22
Chamber size:	17 l	22 l
Power supply:	200–240 V AC; 50/60 Hz; 10 A	
Power consumption:	2.0–2.4 kW	
Overall dimension (w × h × d):	465 × 452 × 634 mm	
Weight (empty):	46 kg	47.5 kg
Main/used water tanks:	4.8/4.8 l	
Working range:	7 to 15 cycles	
Usable space in chamber (w × h × d) ⁽¹⁾ :	195 × 195 × 312 mm	195 × 195 × 400 mm
Connection types:	5 USB ports, 1 Ethernet port, integrated automatic water filling connection	

(1) Usable space is referred to standard rack configuration

Lara XL

CE₀₀₁ CE₀₀₇

Trade name: Type:	Lara XL RIS-311
Chamber size:	28 l
Power supply:	200–240 V AC; 50/60 Hz; 10 A single phase
Power consumption:	2.0–2.4 kW
Overall dimension (w × h × d):	492 mm × 455 mm × 614 mm
Weight (empty):	54 kg
Main/used water tanks:	4.8 l/5.1 l
Working range:	5 to 10 cycles
Usable space in chamber (w × h × d) ⁽¹⁾ :	230 × 230 × 420 mm
Connection types:	5 USB ports, 1 Ethernet port, automatic water filling kit

(1) Usable space is referred to standard rack configuration

Lara



Trade name: Type:	Lara 17 RIL-117	Lara 22 RIL-122
Chamber size:	17 l	22 l
Power supply:	200–240 V AC; 50/60 Hz; 10 A	
Power consumption:	2.0–2.4 kW	
Overall dimension (w × h × d):	465 × 452 × 646 mm	
Weight (empty):	42.5 kg	44 kg
Main/used water tanks:	4.8/4.8 l	
Working range:	7 to 15 cycles	
Usable space in chamber (w × h × d) ⁽¹⁾ :	195 × 195 × 312 mm	195 × 195 × 400 mm
Connection types:	2 USB ports, integrated automatic water filling connection	

(1) Usable space is referred to standard rack configuration

Lina



Trade name: Type:	Lina 17 RIK-117	Lina 22 RIK-122
Chamber size:	17 l	22 l
Power supply:	200–240 V AC; 50 Hz or 60 Hz; 10 A single phase	100–125 V AC; 50 Hz/60 Hz; 12 A single phase 200–240 V AC; 50 Hz or 60 Hz; 10 A single phase
Power consumption:	2.0–2.4 kW	110 V: 1.2–1.5 kW 220 V: 2.0–2.4 kW
Overall dimension (w × h × d):	465 × 452 × 646 mm	
Weight (empty):	40 kg	42 kg
Main/used water tanks:	4.8/4.8 l	
Working range:	7 to 10 cycles	
Usable space in chamber (w × h × d) ⁽¹⁾ :	195 × 195 × 312 mm	195 × 195 × 400 mm
Connection types:	1 USB port in the front (additional 1 in the rear as option), automatic water filling kit (optional)	

(1) Usable space is referred to standard rack configuration


Lyla



Trade name: Type:	Lyla 17 RIN-210	Lyla 22 RIN-210
Chamber size:	17 l	22 l
Power supply:	200–240 V AC; 50 Hz or 60 Hz; 10 A single phase	100–125 V AC; 50 Hz/60 Hz; 12 A single phase 200–240 V AC; 50 Hz or 60 Hz; 10 A single phase
Power consumption:	2.0–2.4 kW	110 V: 1.2–1.5 kW 220 V: 2.0–2.4 kW
Overall dimension (w × h × d):	465 × 452 × 646 mm	
Weight (empty):	40 kg	42 kg
Main/used water tanks:	4.8/4.8 l	
Working range:	7 to 10 cycles	
Usable space in chamber (w × h × d) ⁽¹⁾ :	195 × 195 × 312 mm	195 × 195 × 400 mm
Connection types:	1 USB port in the front (additional 1 in the rear as option), automatic water filling kit (optional)	

(1) Usable space is referred to standard rack configuration

Lexa

Meets the requirements of ASME Boiler and Pressure Vessel Code 

Trade name: Type:	Lexa 110 MN-111	Lexa 220 MN-111
Chamber size:	28 liters (7.4 gal)	
Power supply:	100–125 V AC; 50/60 Hz; 12 A Single Phase	200–240 V AC; 50/60 Hz; 10 A Single Phase
Overall dimension (w × h × d):	490 × 452 × 616 mm (19.3" × 17.8" × 24.3")	
Weight (empty):	48 kg (105.8 lbs)	
Main/used water tanks:	6.4/6.7 l (1.7/1.8 gal)	
Connection types:	5 USB ports, 1 Ethernet port	

EliReel

EliReel							
Width × length:	5.5 cm × 200 m	7.5 cm × 200 m	10 cm × 200 m	15 cm × 200 m	20 cm × 200 m	25 cm × 200 m	30 cm × 200 m
Paper:	60g/m ²						
Plastic:	PET/PP (polyester and polypropylene)						
Duration of sterility:	180 days (6 months) when stored correctly (temperature 20–25°C and humidity 35–50%)						

Seal²

CE

Seal ²		
Voltage:	200/240 V AC; single phase	100/120 V AC; single phase
Frequency:	50/60 Hz	
Power:	180 W	
Isolation class:	I/IPX0	
Fuse:	N. 2-5x 20 T4 A	N. 2-5x 20 F10 A
Conditions of use:	Indoor use only	
Environment:	3000 m a.s.l.	
Max. altitude:	+5/+40°C	
Temperature:	85 %	
Max. relative humidity:	±10 %	
Max. voltage tolerance:	II	
Overvoltage category:	1	
Pollution degree:	Bipolar	
Main switch:	7.1 kg	
Weight:	12 mm × 310 mm	
Welding area:	Constant	
Welding temperature:		

SealVal²

CE

SealVal ²		
Voltage:	200/240 V AC; single phase	
Frequency:	50/60 Hz	
Power:	180 W	
Isolation class:	I/IPX0	
Fuse:	Nr. 2-5x 20 T5 A	
Conditions of use:	Indoor use only	
Environment:	3000 m a.s.l.	
Max. altitude:	+5/+40°C	
Temperature:	85 %	
Max. relative humidity:	±10 %	
Max. voltage tolerance:	II	
Overvoltage category:	2	
Pollution degree:	230 V + 1200 V	
Max. surge (≤ 5s):	230 V + 250 V	
Max. surge (> 5s):	Bipolar	
Main switch:	8.3 kg	
Weight:	12 mm × 310 mm	
Welding area:		
Welding temperature:	Standard mode Free option mode	180°C min. 170°C max. 200°C (set by the user)

Multidem, Osmo, Water Distiller

CE




CE

	Multidem	Osmo	Water Distiller
Water treatment principle:	Demineralization	Reverse Osmosis	Distillation
Direct connection to sterilizer:	yes	yes	no
Number of connectable sterilizers:	up to 2 units	up to 6 units	-
Possibility of manual filling:	yes (spray gun)	yes (spray gun)	yes
Fulfilling sterilization requirements:	yes	yes	yes
Water quality output:	0.5-2 µS/cm	0.5-5 µS/cm	5-10 µS/cm
Produced water:	60 litres/1h	30 litres/1h	4 litres/5h
Volume storage tank:	-	7 litres	4 litres
Water pressure (min./max.):	2-8.6 bar	0.5-5 bar	-
Main device dimensions (w x h x d):	12,4 × 12,3 × 47,6 cm	37 × 52 × 13 cm	23 cm (Ø) × 36 cm (h)
Water tank dimensions (w x h x d):	-	23 cm (Ø) × 35 cm (h)	17 cm (Ø) × 25 cm (h)
Weight:	2.7 kg	13 kg	3.5 kg
Electrical supply needed:	no	yes	yes
Electrical power supply:	-	230 V AC; 50/60 Hz; 0.6 A 115 V AC; 50/60 Hz; 1.2 A	230 V AC; 50 Hz
Wall mount:	no	yes	no
Backflow preventer according to EN1717:	yes	yes	-
Demineralized water according to EN13060 Annex C:	yes	yes	yes

W&H adaptor set for washer disinfectors

Compact, practical, universal.

The W&H adaptor set enables uncomplicated, validated mechanical reprocessing of accessories and small parts in the washer disinfector – internal and external. Instruments available again.

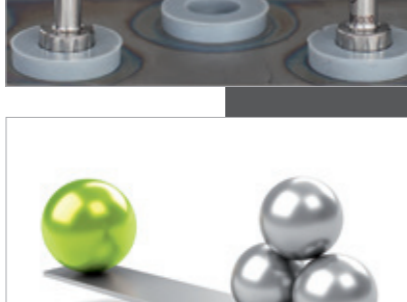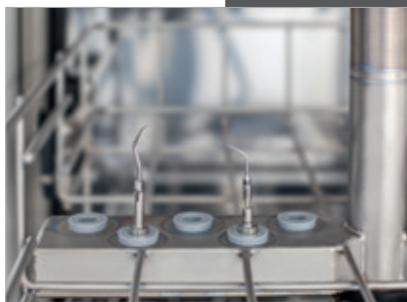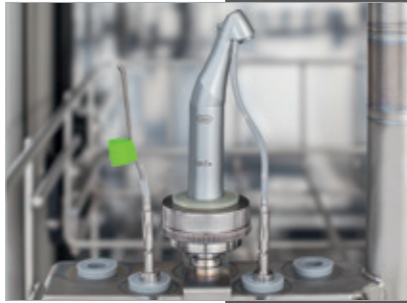

W&H adaptor set

The W&H adaptor set was specially designed for mechanical reprocessing of accessories and small parts.

Validated cleaning

Supports validated internal cleaning and disinfection of the hollows of instruments such as scaler tips or coolant tubes.

Internal and external thread cleaning

A special construction of the adaptor head guarantees that internal and external threads are cleaned.

Universal application

Usable with all rinsing channels in the washer disinfector with an internal thread of 8 mm (M8x1) or a matching rubber insert. The W&H adaptor set is compatible with all tips with M3x0.5 mm thread. For a complete list see wh.com.

Value for money

Maximum cleaning and disinfection efficiency at a low purchase price.

System overview adaptors Assistina Twin & Assistina One

Straight and contra-angle handpieces

All straight and contra-angle handpieces with ISO connection

Contra-angles with removable heads

T1 CLASSIC from Sirona Dental Systems for contra-angle handpieces

FG chuck system (W&H, Sirona Dental Systems)

W&H handpiece and contra-angle head



Adaptor REF 02679000



Adaptor REF 02691000



Adaptor REF 02693000



Adaptor REF 05204600



Adaptor Quick RM REF 06395100



Turbines

W&H turbines, air scaler and air motors with Roto Quick system

T1, T2, T3 from Sirona Dental Systems

Turbines with Multiflex connection from Kaltenbach & Voigt

Turbines with Borden 2(3)-hole connection

Turbines or adaptors with standard 4-hole connection (with or without light)



W&H Adaptor REF 02690400



Adaptor REF 02692000



Adaptor REF 04713200



Adaptor REF 02083500

Adaptor Quick RM REF 06395100



Distribution:

W&H Dentalwerk Bürmoos GmbH

Ignaz-Glaser-Straße 53, Postfach 1

5111 Bürmoos, **Austria**

t +43 6274 6236-0

f +43 6274 6236-55

office@wh.com

wh.com

W&H Sterilization Srl.

Via Bolgara, 2

24060 Brusaporto (BG), **Italy**

t +39 035-66 63 000

f +39 035-50 96 988

office.sterilization@wh.com

wh.com