



## Tyscor VS 1 / VS 2 combination suction unit



Order number: 7186100001



Ambient temperature



Relative humidity




Only qualified specialists or employees trained by Dürr Dental are permitted to install, connect and start using the unit.




## Installation


1.

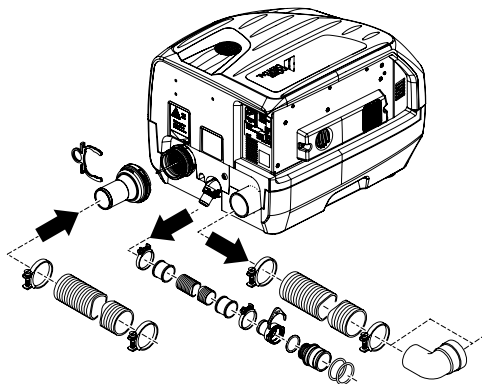

2.

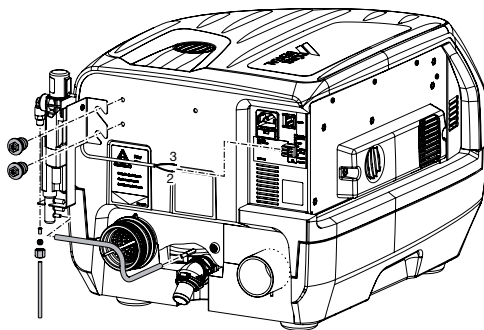

3.

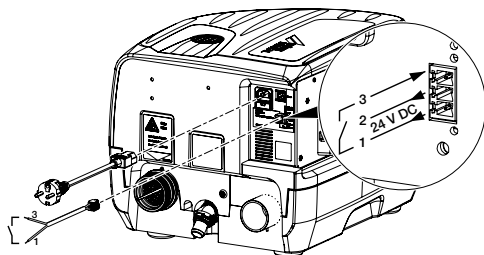

**DÜRR DENTAL SE**

Höpfigheimer Str. 17  
74321 Bietigheim-Bissingen  
Germany

9000-606-130/01




1701V003SE

**DÜRR  
DENTAL**

# Operation


1.

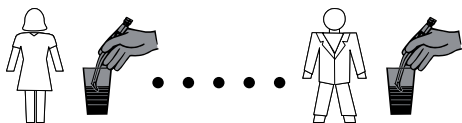


- After treating each patient, aspirate one glass of cold water.

2.

**1x / Day in the evenings**

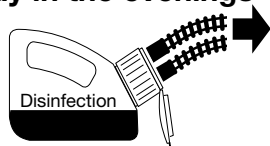


**1x / Week about noon**

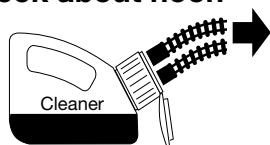


- Daily, after completing treatment, aspirate a unit of disinfection solution, e.g. Orotol plus.
- Once a week before the lunch break, aspirate a unit of cleaning agent solution, e.g. MD 555 cleaner. Rinse with water after the reaction time.

3.

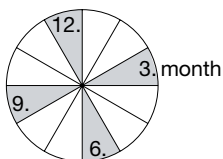

1.

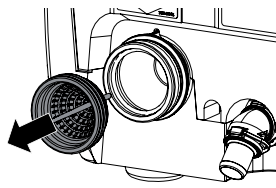

2.



or

**NEW**



- Clean or replace the protective sieve on the suction input every 3 months.