

# A strong team for the suction unit

## Daily Orotol® plus

With fluid-concentrate Orotol® plus, you have the ideal solution for simultaneous disinfection, deodorisation, cleaning and maintenance of all suction systems and amalgam separators. Application is done after treatments are complete, at high force 2 x daily (noon and evening).

1. For preliminary treatment: Draw off 2 litres of water
2. Apply the solution in the care system.\*  
For a treatment unit: Pour in 20 ml (1 dosing cap), fill to 1 litre with cold water and mix thoroughly (double the quantities for 2 treatment units)
3. Attach suction hoses to the care system and suck up solution
4. Apply another 1 litre and distribute to max. four spittoons
5. Allow to react for at least 1 hour or overnight
6. After the reaction time/next morning, suck up 2 litres of water



## Weekly MD 555 cleaner

MD 555 cleaner is your special cleaner for suction units and amalgam separators. It has powerful cleaning forces against deposits, scaling and siltation. It is applied 1 to 2 x weekly (e.g. Tuesday and Thursday), with higher build-up, daily before noon-break.

1. For preliminary treatment: Draw off 2 litres of water
2. Apply the solution in the care system.\*  
For a treatment unit: Pour in 50 ml (2.5 dosing caps), fill to 1 litre with cold water and mix thoroughly (double the quantities for 2 treatment units)
3. Attach suction hoses to the care system and suck up solution
4. Apply another 1 litre and distribute to max. four spittoons
5. Allow to react for at least 0.5 to 2 hours (noon-break)
6. After the reaction time, suck up 2 litres of water

\* Using a care system with suction hose adaptors to suck an air-solution mixture. Sucking without the respective adapter from the bowl or other container is not suitable for disinfection and cleaning. We recommend using the OroCup.